BORT Volare Lagerungsschiene

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REF 103 100

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D103100|2023-07|005 ML | Rev. 01

DE BORT Volare Lagerungsschiene

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EN BORT Volar Resting Splint

Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

Intended purpose

This medical device is a wrist brace with a finger and thumb rest for immobilisation.

Indications

Pre-operative, post-operative, post-traumatic, degenerative, e.g. carpal tunnel syndrome (CTS), for paralysis, for inflammatory and rheumatic diseases, distortion.

Contraindications

Peripheral arterial occlusive disease (PAOD), lymph drainage disorders, also unclear soft tissue swellings distal to the aid positioned, sensory and circulatory disorders with the region of the body treated, skin diseases in the part of the body treated.

Application risks/Important notes ⚠

- This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.
- remove the medical device prior to radiological examinations
 - do not close it too tightly during longer periods of rest, e.g. when sleeping
 - in case of numbness, loosen the medical device and remove it if necessary
 - in case of persistent complaints, consult the physician or specialist retailer
 - use the medical device in accordance with therapeutic needs
 - only use other products simultaneously after consultation with your physician
 - do not make any changes to the product
 - do not wear it on open wounds
 - do not use in case of intolerance of one of the materials used
 - no re-use. This medical aid is intended for treating one patient only
 - whilst wearing the brace, please neither use any creams nor ointments in the vicinity of the attached aid as they can destroy the material

Fitting instructions

Place the brace on the inner side of the hand/lower arm. The aluminium splint is mouldable, enabling achievement of the desired thumb and hand position. Place the finger on the finger rest and place the thumb in the thumb guide. Firstly, close the Velcro fastener on the metacarpus. Finally, close the Velcro fastener on the wrist and lower arm.

Removal

Open all the Velcro fasteners and remove the brace. Close the Velcro fasteners in order to avoid early wear to the Velcro surface.

Material composition

Polyethylene foam (PE), polyamide (PA), elastane (EL)
The sewn in textile label provides the precise material composition.

Cleaning information

☑️ Hand-washing 30 °C ☒ Do not bleach ☑️ Do not dry in a tumble dryer ☒ Do not iron ☒ Do not clean chemically
Close the Velcro fasteners in order to avoid damaging other items of clothing. Do not use fabric conditioner.

Guarantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

Useful life/Lifetime of the product

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

Duty of notification

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA). You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: www.bort.com/md-eu-contact.

Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: www.bort.com/conformity

Status: 07.2023

MD Medical device | Single patient – multiple use

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