ANLEGEN DER MALLEODYN® S3

FITTING THE MALLEODYN® S3

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IMPORTANT INFORMATION

Use the MALLEODYN® S3 immediately after injury or during the acute phase in its fully assembled state plus the cooling pad. The orthosis should always be worn over a sock.

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STEP 1

Open the figure 8 strap at the two rear Velcro ends and the orange-coloured click fasteners on the lower ankle and foot part.

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STEP 2

Place your foot at a 90° angle in the open orthosis. Position your foot so that the outer ankle lies centrally in the cut-out of the side splint.

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STEP 3

Position the included cooling pad from the inside between the sock and the MALLEODYN® S3. The position can be varied, depending on your injury.

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USING THE COOLING PAD

- Position the cooling pad, with the textile side facing the skin, anywhere within the orthosis between the sock and the orthosis. We recommend intermittent cooling during the acute phase. To cool it, place the cooling pad only in the refrigerator, not in the freezer.
- Please consult with the doctor in charge of your care for advice on how to use the cooling pad.

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STEP 4

Click in the orange foot closure on the inside of the foot and tighten the strap with the Velcro fastener.

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STEP 5

Close the click closure on the lower ankle by inserting the orange tab into the plastic shell, folding it backwards and then click the push-button into the plastic shell. If necessary, adjust the tension of the lower ankle strap using the upper Velcro fastener.

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STEP 6

Close the figure 8 strap by crossing them over the back of the foot and pulling them upwards towards the calf, where you can secure them in place using the Velcro ends.

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DISMOUNTING THE MALLEODYN® S3 IN 3 STEPS

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STEP 1

Remove the cooling pad from the fully assembled brace.

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STEP 2

Remove the side splint with a click. To do this, press on the orange outer strip and pull the splint upwards and out.

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STEP 3

Remove the figure 8 strap attached at the front edge by loosening the Velcro fastening.

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ADVICE

The lateral reinforcement splint and the figure 8 strap, once removed, can be stored in the wash bag supplied if you wish to reuse them.