EN Smart Digital Thermometer
Eco Temp Intelli IT (MC-280B)
Instruction Manual

All for Healthcare

1. Introduction

Thank you for purchasing the OMRON Digital Thermometer Eco Temp Intelli IT. This thermometer eliminates any worries about broken glass or mercury hazards. Note that your temperature is affected by smoking, eating or drinking. Contact your doctor if you have any questions regarding specific temperatures.

Intended Use
The thermometer you purchased offers a safe, accurate and quick temperature reading. You can measure your temperature either in the mouth (orally) or underarm (axillary).

Intended Operator
This thermometer is intended to be operated by adults who can understand this instruction manual. It is mainly designed for household use.

2. Important Safety Information

Read the Important Safety Information in this instruction manual before using this device. Follow this instruction manual thoroughly for your safety. Keep for future reference.

Warning Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

- Keep the thermometer out of the reach of infants, toddlers and children. DO NOT allow children to operate/ use the thermometer while unattended. Children may injure themselves or other children when attempting to take temperatures without appropriate supervision.
- To help avoid choking, keep the battery, battery cover and screw away from infants, toddlers and children. If any of these are swallowed, consult with your physician immediately.
- NEVER diagnose or treat yourself based on your temperature readings. ALWAYS consult with your physician.
- DO NOT use the thermometer to measure anything other than human body temperature.
- This thermometer is designed for oral and underarm temperature measurements only. DO NOT attempt to take temperature at other sites, such as in the ear or rectum, as it may result in false readings and may lead to injury.
- DO NOT attempt to take temperature measurements when the thermometer is wet, as this may cause inaccurate readings.
- DO NOT use the device with other medical electrical (IME) equipment simultaneously. This may result in incorrect operation of the device and/or cause an inaccurate reading.

Battery Handling and Usage
DO NOT throw the thermometer into fire.

Caution Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient, or cause damage to the equipment or other property.

- This thermometer does not have water-resistant functionality. DO NOT immerse the thermometer in water.
- DO NOT use the thermometer in places where strong static electricity or electromagnetic fields are present. It may cause an inaccurate measurement result and/or cause the thermometer to operate incorrectly.
- DO NOT allow the thermometer to be shared among individuals without properly disinfecting the device.
- DO NOT disassemble or attempt to repair the thermometer. It may cause inaccurate reading and/or result in incorrect operation of the thermometer.
- DO NOT bite on the thermometer. It may cause breakage and/or injury.
- DO NOT pull or bend the thermometer.
- DO NOT use this thermometer in places with high or low humidity or high or low temperatures. Refer to section 9.
- DO NOT drop the thermometer or storage case or subject them to strong shocks or vibrations.
- During measurement, make sure that no mobile device or any other electrical devices that emit electromagnetic fields is within 30 cm of this thermometer. This may result in incorrect operation of the thermometer and/or cause an inaccurate reading.


Battery Handling and Usage
DO NOT insert the battery with its polarity incorrectly aligned.
ONLY use a CR1220 lithium button battery with this thermometer. DO NOT use any other type of battery.
Remove the battery when the thermometer will not be used for 3 months or more.
If battery fluid gets in your eyes, immediately rinse with plenty of clean water. Consult with your physician immediately.
DO NOT replace the battery while the power is on.
DO NOT use the battery after its expiration date.

3. Measuring Temperature Correctly

Measurement accuracy cannot be guaranteed when the thermometer is incorrectly used to take a temperature measurement.

Oral use (for ages 4 and older):
The mouth should remain closed up to 5 minutes before measurement.
Place the thermometer in the mouth under the tongue so that it rests to the left or right of the root of the tongue.
Use downward tongue pressure to hold the thermometer in place.
Hold the thermometer to keep it from sliding around in the mouth.

- Open the transfer screen of the app on your smartphone.
- While "°" symbol flashes, press the power button.
The thermometer will be ready for transferring your temperature measurement result.
Note
 - Your temperature measurement result cannot be transferred after the thermometer is turned off.
 - When the thermometer is ready for transferring, you may hear noise from the thermometer. It is not a malfunction.
 - It is recommended to transfer your temperature measurement result in a quiet place.

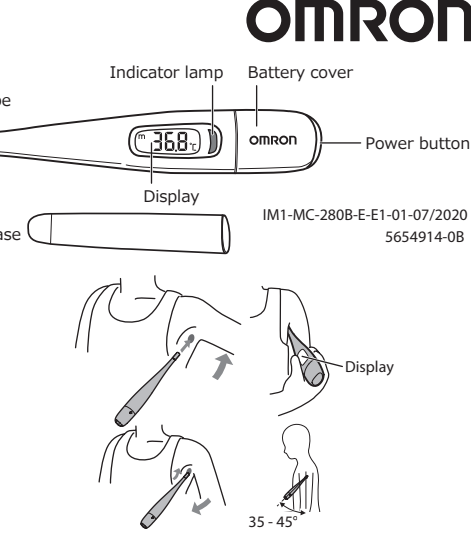- The following may result in incorrect measurements:**
- Taking the temperature measurement immediately after exercising, bathing, eating or drinking.
 - Do: Wait at least 30 minutes before taking the temperature.
 - Taking a temperature measurement after moving around after awakening.
 - Do: Upon waking it is recommended to take a measurement before starting your morning routine because this can increase your body temperature. It is recommended to measure directly after waking up or rest 30 minutes before taking a measurement.
 - Taking a measurement repeatedly.
 - Do: Turn the power off and wait at least 30 seconds, and then take a temperature again.
 - Taking a measurement while your underarm is sweating.
 - Do: Wipe off any underarm perspiration before taking the temperature.
 - Taking a measurement after being under the blankets for a long time.
 - Do: Cool down the heat, then take a temperature.
 - The sensor tip is in contact with clothing.
 - Do: Place the sensor tip in the center of the underarm and ensure that it only comes into contact with the skin, not clothing during measurement.

4. Using Thermometer

To transfer your measurements/readings to your smartphone, first download and install the "OMRON connect" app and follow the instructions within the app.

- Press the power button.
1. All symbols appear on the display as an internal test function, the indicator lamp lights and the buzzer beeps.
The battery symbol appears every time the power is turned on. It is not a sign of the battery replacement.
- The most recent temperature measurement result with "m" is displayed.
- When "°C" ("°F") flashes, the thermometer is ready for use.
Note
 - The display will show "L" (for the actual temperature if it is stored in warm ambient environment above 32.0 °C).

- Apply the thermometer to the measurement site.
Refer to section 3 to apply the thermometer correctly.
- The buzzer will emit (beep-beep-beep) sound 3 times when the temperature becomes stable.
"°" symbol and indicator lamp flash.
The displayed body temperature at this timing is just the early indication of body temperature, so it may slightly differ from the actual body temperature.
To know your actual body temperature, keep applying the thermometer for the specified time as "Actual temperature measurement time".
If you want to know your body temperature earlier, remove the thermometer from the measurement site and read the result.

- Buzzer timing
Oral: approx. 40 seconds
Underarm: approx. 40 seconds
The buzzer timing is based on an environmental temperature of 23 °C (73.4 °F).
- Actual temperature measurement time
Oral: approx. 5 minutes
Underarm: approx. 10 minutes
The buzzer will NOT beep even if the time above has elapsed. Measure the time yourself and remove the thermometer when the time above has elapsed.
- Open the transfer screen of the app on your smartphone.
- While "°" symbol flashes, press the power button.
The thermometer will be ready for transferring your temperature measurement result.
Note
 - Your temperature measurement result cannot be transferred after the thermometer is turned off.
 - When the thermometer is ready for transferring, you may hear noise from the thermometer. It is not a malfunction.
 - It is recommended to transfer your temperature measurement result in a quiet place.

Go to next column for the next step

6. Place the thermometer on your smartphone's microphone as the illustration to the right.

- The transmission will start automatically and will continue transferring the measurement reading to the app for the next 2 minutes until the thermometer is turned off.
Note
 - For further details regarding transferring your temperature readings to your smartphone, refer to the transfer screen of the app.
 - Check within the app to make sure your temperature is successfully transferred before turning off the thermometer.
 - If you feel any discomfort from the thermometer's sound, turn off the thermometer immediately after the data transfer.

- Press the power button to turn off the thermometer.
Clean and disinfect the thermometer before returning it to the storage case. Refer to sub-section 7.2.

- 5. Replacing Battery**
Battery: CR1220 lithium button battery (commercially available)
Use the battery within the recommended period indicated by the "A" manufacturer.
Replace the battery if " " and "A1" appear when the thermometer is turned on.
- Remove the battery cover with a small cross-slot screwdriver.
 - Remove the battery with a thin object such as a pen.

- Note**
When removing the battery, be careful not to injure yourself as the battery may jump out.
- Insert new battery with the "+" pole facing up as shown in the following figure.

- Replace the battery cover and secure it with the screw.

- 6. Switching between "°C and "°F**
- While the power is off, press and hold the power button for 15 seconds until "°" appears.
After switching, the power will automatically turn off. To switch to "°C, do the same step above again.

- Maintenance
- 7.1 Storage**
Store the thermometer in the storage case.
Do not store the thermometer:
In locations subject to the water because the thermometer is not waterproof;
In locations exposed to extreme temperatures, humidity, close to heating equipment, direct sunlight, dust or corrosive vapors such as bleach;
In locations exposed to vibration or shock.
- 7.2 Cleaning and Disinfecting**
When disinfecting, use a cotton swab moistened with alcohol-based sanitizer and wipe the thermometer 2 or 3 times.
When using alcohol to clean the thermometer, make sure that it does not come into contact with the indicator section.
Wipe the thermometer with a soft clean cloth.
For stubborn stains, wipe using a soft cloth dampened with lukewarm water then wipe the area with a dry cloth.
Observe the following to prevent damage to the thermometer:
Do not use benzene, thinner, gasoline or other strong solvents to clean the thermometer;
Do not soak the sensor tip in alcohol for long periods of time to attempt to sterilize it using hot water (water at a temperature of 50 °C (122 °F) or higher).

- 8. Error Messages and Troubleshooting**
If any of the below problems occur during measurement, check to make sure that no other electrical device is within 30 cm. If the problem persists, please refer to the table of this section.
- | Display/Problem | Possible Cause | Solution |
|---------------------------------|---|--|
| Any of 0 - 4 appears with "Er". | The thermometer may have a malfunction. | Contact your local OMRON representative. |

- | Display/Problem | Possible Cause | Solution |
|--|---|---|
| "AH-" appears. | The thermometer is/was stored in a location where the temperature is/was 40 °C (104 °F) or more. | Take a temperature measurement immediately after leaving the thermometer in a location where the temperature is between 10 and 40 °C (50 and 104 °F) for at least 1 minute. |
| "AL-" appears. | The thermometer is/was stored in a location where the temperature is/was lower than 10 °C (50 °F). | Take a temperature measurement immediately after leaving the thermometer in a location where the temperature is between 10 and 40 °C (50 and 104 °F) for at least 1 minute. |
| When the thermometer is turned on "L" appears. | The temperature of the sensor tip is lower than 32 °C (89.6 °F). | This is not a malfunction. Keep taking a temperature measurement. |
| When the thermometer is turned on "H" appears. | The temperature of the sensor tip is higher than 42 °C (107.6 °F). | Cool the sensor tip with a damp towel and then take a temperature measurement. |
| When the thermometer is turned on a random reading between 32.0 to 42.0 °C (89.6 to 107.6 °F) appears. | The sensor tip is warm and the display shows the actual ambient temperature, if the ambient temperature is higher than 32 °C (89.6 °F). | |
| "Ca" flashes any time. | Battery is low. | Replacing the battery with new one is recommended. Refer to section 5. |
| Nothing appears on "Ca" and "A1" appear on the display when the power button is pressed. | Battery polarity is not properly aligned. | Check the battery installation for proper placement. Refer to section 5. |
| Battery is depleted. | Battery is depleted. | Immediately replace new battery. Refer to section 5. |
| Difference in temperature between 2 consecutive measurements or between different thermometers. | Complete the measurements for the full measurement time. Different measurement times can lead to minor differences between measurement results. | |
| The temperature readings are inconsistent. | The thermometer is being placed at a different measurement site for each measurement. | Apply the thermometer correctly. Refer to section 3. |
| Transferring the temperature is not successfully completed. | The thermometer is too far from your smartphone's microphone.
The environment during the transfer of the temperature is too noisy. | Bring the thermometer closer to your smartphone's microphone.
Transfer your temperature in a quieter place. |

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Product category	Clinical Electronic Thermometers
Product description	Smart Digital Thermometer
Model (code)	Eco Temp Intelli IT (MC-280B)
Sensing unit	Thermistor
Measurement accuracy	±0.1 °C (32.0 to 42.0 °C) / ±0.2 °F (89.6 to 107.6 °F) (when measured at a standard room temperature of 23 °C (73.4 °F) in a test tank held at constant temperature)
Temperature display	3 digits (°C), or 4 digits (°F) display in 0.1 degree increments
Measurement position	Oral, underarm
Measurement time	Underarm: approximately 10 minutes; Oral: approximately 5 minutes
Power supply	3 V DC, CR1220 lithium button battery
Durable period (service life)	5 years
Battery life	Approx. 1.5 years or more (1 time per day)
Measurement method	Actual temperature measurement (non-predictive)
Operating mode	Continuous operation
Applied part	Type BF (sensor tip and probe)
Protection against electric shock	Internally powered ME equipment
Measurement range	32.0 to 42.0 °C (89.6 to 107.6 °F)
Memory	1 Memory
Operating conditions	+10 to +40 °C (+50 to +104 °F), 10 to 85% RH (non-condensing), 700 - 1060 Pa
Storage/Transport conditions (between uses)	-20 to +60 °C (-4 to +140 °F), 10 to 90% RH (non-condensing)
Weight	Approx. 14 g (with battery installed)
Dimensions	Approx. 20.1 mm (w) x 36.8 mm (l) x 12.8 mm (d)
Package contents	Main unit, CR1220 lithium button battery, storage case, instruction manual

- The specifications may be changed without prior notice.
- The sensor tip of this thermometer contains nickel.
- This thermometer does not contain any servicable part. OMRON recommends that the accuracy is verified every two years by an OMRON representative.
- Operation mode classification is in accordance with IEC 60601-1.
- This device fulfills the provisions of the European Standard EN12470-3:2000+A1:2009, Clinical thermometers - Part 3: Performance of compact electrical thermometers (non-predictive and predictive) with maximum device.
- Please report to the manufacturer and the competent authority of the Member State in which you are established about any serious incident that has occurred in relation to this device.

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Guidance and Manufacturer's Declaration

Anleitung und Herstellererklärung/Conseils et déclaration du fabricant/Guida e dichiarazione del produttore/Guia y declaración del fabricante/Richtlijnen en verklaring van de fabrikant/Rehberlik ve Üretici Beyanı/Указания и заявление производителя/Vejledning og producentdeklaration/Objekt y valmistajan ilmoitus/Veljedning og produsentetklärung/Riktlinjer och tillverkarens deklaration/Οδηγίες και δήλωση κατασκευστή/Orientação e declaração do fabricante/ Wskazówki i deklaracja producenta/الإرشاد وإعلان الشركة المصنعة

EN
Important information regarding Electromagnetic Compatibility (EMC)
MC-280B-E manufactured by OMRON HEALTHCARE Co., Ltd. conforms to EN60601-1-2:2015 Electromagnetic Compatibility (EMC) standard. Further documentation in accordance with this EMC standard is available at OMRON HEALTHCARE EUROPE at the address mentioned in this instruction manual or at www.omron-healthcare.com. Refer to the EMC information for MC-280B-E on the website.

Correct Disposal of This Product
This marking shown on the product or its literature, indicates that it should not be disposed of, with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources.

Waste Electrical & Electronic Equipment
Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can take this item for environmentally safe recycling. Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial wastes for disposal.

Battery
Disposal of used batteries should be carried out in accordance with the national regulations for the disposal of batteries.

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