|                                        |             |    |
|----------------------------------------|-------------|----|
| Selbstklebendes Kondom-Urinal          | Deutsch     | 2  |
| Self-Adhesive Urinary Sheath           | English     | 4  |
| Étui pénien autocollant                | Français    | 6  |
| Condom urinario autoadesivo            | Italiano    | 8  |
| Zelfklevende externe katheters         | Nederlands  | 10 |
| Självhäftande urindroppssamlare        | Svenska     | 12 |
| Selvklebende urinalkondom              | Dansk       | 14 |
| Itseliimautuva urinaalikondomi         | Suomi       | 16 |
| Selvimende uridom                      | Norsk       | 18 |
| Isekleepuv uriinikogur                 | Eesti       | 20 |
| Colector de orina peneano autoadhesivo | Español     | 22 |
| Preservativo urinário autoadesivo      | Português   | 24 |
| Αυτοκόλλητος περιπεϊικός καθετήρας     | Ελληνική    | 26 |
| Öntapadó vizeletkondom                 | Magyar      | 28 |
| Samoprzylepny cewnik zewnętrzny        | Polski      | 30 |
| Samolepicí urinální kondom             | Čeština     | 32 |
| Prezervativ urinar autoadeziv          | Română      | 34 |
| Самозалепващ уринален кондом           | Български   | 36 |
| Samolepilni urinal kondom              | Slovenščina | 38 |
| Samoljepivi urinarni kondom            | Hrvatski    | 40 |
| Samolepljivi urinarni kondom           | Srpski      | 42 |
| Samoljepljivi urinarni kondom          | Bosanski    | 44 |
| Самолеплив уринарен кондом             | Македонски  | 46 |

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## Self-Adhesive Urinary Sheath – SAUER Comfort

## Ready for use, for single use

## Contents

- 30 **urinary sheaths**, self-adhesive, made of synthetic material (without addition of latex), powdered, manufactured as one piece with the connecting tube, connects to all leg bags and bed bags with and without adapter

| Order no.  | Colour | length of sheath | buffer zone <b>P</b> | adhesive zone <b>K</b> | adhesive |
|------------|--------|------------------|----------------------|------------------------|----------|
| 97.xx      | white  | 130 mm           | 75 mm                | 50 mm                  | normal   |
| 97.7501.xx | blue   | 130 mm           | 75 mm                | 50 mm                  | strong   |
| 97.6000.xx | green  | 115 mm           | 60 mm                | 50 mm                  | normal   |
| 97.6001.xx | red    | 115 mm           | 60 mm                | 50 mm                  | strong   |
| 97.4511.xx | violet | 75 mm            | 45 mm                | 25 mm                  | strong   |

11 sizes: Ø 18, 20, 22, 24, 26, 28, 30, 32, 35, 37, 40 mm  
 ("xx" in the order no. represents the size of the sheath, e.g.  
 97.30 = 30 mm)

- 1 **protective cloth**, matching the size of the sheath

## Intended use

SAUER Comfort urinary sheaths are used for external urinary drainage into a urine collection bag in men with urinary incontinence or if external urinary drainage is desired for other reasons. They are used on intact skin.

## Indications

Urinary incontinence of varying causes without residual urine (e.g., neurogenic urinary dysfunctions, hyperactive bladder, urinary incontinence after surgery); urinary drainage in people who cannot manage to go to the toilet by themselves.

## Contraindications

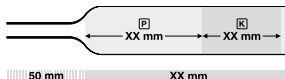
Chronic urinary retention with high intravesical pressure; sphincter dyssynergia with significant urinary retention; constrictions in the urethra (e.g., prostatic hyperplasia) resulting in high residual amounts of urine; skin abrasions on the penis (e.g., wounds, eczema, pressure sores); acute skin diseases (e.g., mycosis, bacterial skin infections); allergies/incompatibilities to the material or adhesive/adhesive spray/adhesive tape; pronounced phimosis.

## Important information on hygiene and period of wear

When using continence products, the hygiene guidelines of the national health authorities must be observed. We recommend changing the sheath (the adhesive tape, bag) after 24 hours. The period of wear may also be less than 24 hours depending on the skin type.

## Selecting the right size/product

Sauer Comfort Urinary Sheaths are available in 5 versions, each with up to 11 sizes. It is essential to select the right version and size so that

the sheath fits properly and does not cause leaks or pressure sores. To select the size, the circumference of the penis must be measured around the middle of the shaft. If it is adhered during erection (see application), the circumference is also measured on the erect penis. The product-specific measuring tapes state the diameter based on the circumference measurement. If leaks or pressure sores occur, the sheath size must be checked and adjusted if necessary.

## Risks and warnings

Ask your doctor if urinary sheaths are suitable for your incontinence care. Instructions on how to use urinary sheaths should be provided by a qualified nurse.

The use of urinary sheaths may cause skin irritation and skin changes. When removing a urinary sheath, inspect the skin. If you notice any abnormalities, stop using the product and seek medical advice.

If urinary sheaths are worn too tightly, this may cause pressure sores and, in very rare cases (particularly in the case of lack of sensitivity), constrictions and necrosis. In this case, seek a doctor without delay.

## Application

## Preparations

Wash the skin in the whole genital area thoroughly and dry well. If used on damp skin or on skin that was pre-treated with oily/greasy products, the adhesive effect is reduced.

An erection is not absolutely necessary for adhesion, but if one can be manipulated, we recommend that you apply the sheath to an erect penis as this gives a more stable surface for adhesion and prevents

constriction.

Optional: Pull over the protective cloth so that the pubic hair does not stick or obstruct adhesion.

### Applying the sheath

Unroll the buffer zone. Depending on the version of the sheath, unroll it 1–3 folds. For example, 1 fold for the violet sheaths, 2 folds for the green and red sheaths, and 3 folds for the white and blue sheaths (Fig. 1). This unrolled area provides the required buffer zone between the penis and the tip of the sheath, as well as the correct position of the adhesive zone. If the sheath is difficult to unroll, it helps to stretch the connecting tube slightly.

Position the sheath on the foreskin without smoothing the foreskin back (Fig. 2). If an erection is not possible, pull the foreskin gently to smooth the skin.

When unrolling further, you reach the adhesive zone (Fig. 3). To make unrolling easier, you can also stretch the connecting tube here. From the start, unroll the sheath **evenly all around**.

Avoid any folds in the adhesive area. Check the position of the sheath in the adhesive area and press it carefully around and against the shaft (Fig. 4). If necessary, rub out longitudinal folds and air bubbles.

### Removing the ring

The remaining ring beyond the adhesive area can be cut open or cut/torn off completely using scissors with rounded tips if you are worried about pressure sores or if the penis retracts considerably, e.g., when urinating (Fig. 5). Do not use scissors with pointed tips as they may cause injury. The incision should be circular in shape so that the sheath does not tear. When cutting or

If the sheath is unrolled unevenly towards one side, it may stick and then be almost impossible to unroll.

tearing off, press down on the adhesive area of the sheath at the same time.

### Connecting to the bag

Push the connecting tube of the sheath on the adapter of the bag and press all around gently. For bags without adapter, push the connecting tube directly on the inlet tube. Make sure that the buffer zone and inlet tube are not twisted or bent.

### Removing the sheath

Remove the sheath after 24 hours at the latest and pull or roll off from the back towards the tip of the penis. Do not pull too strongly so as to avoid skin abrasions. If the sheath adheres too strongly, lift the upper end of the sheath slightly and use an oil pad, for example, to wipe between the sheath and the skin in order to gradually undo the adhesive bond. Carefully remove any adhesive remains

### Disposal

Dispose of the sheath in household waste.

|               |                                 |
|---------------|---------------------------------|
|               | Article number                  |
|               | Store in a dry place            |
|               | Quantity                        |
|               | Keep away from sunlight         |
| <b>CH REP</b> | Swiss authorised representative |
| <b>LOT</b>    | Batch number                    |
|               | Observe instructions for use    |
|               | CE marked medical device        |
|               | Use by (year-month-day)         |
|               | Not for reuse                   |
| <b>PZN</b>    | Pharmaceutical central no.      |
| <b>DT</b>     | Drug Tariff                     |
| <b>LPP</b>    | List of products and services   |
|               | Manufacturer                    |
|               | Importer                        |

If serious incidents occur when using “SAUER Comfort” urinary sheaths, these must be reported to the manufacturer and competent authority of the country in which the user resides.

MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.

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