DERMATOP PATCH HI TECH

EN | DERMATOP PATCH HI TECH

Adhesive dressing soaked in hyaluronic acid sodium salt.
Protects and repairs.

Product information

DERMATOP PATCH HI TECH is a medical device for cutaneous use consisting of a transparent adhesive edge made of polyurethane and a multi-layer absorbent pad in the middle, impregnated with a 0.3% sodium salt hyaluronic acid solution.

The dressing has been developed for a high level of comfort and:

- has rounded edges for optimum fit;
- allows the skin to breathe;
- is waterproof;
- is water-resistant.

The central pad also has a non-stick layer for painless and trauma-free removal.

DERMATOP PATCH HI TECH creates a moist environment at the site of application, protected from the outside, so as to favour the natural process of tissue repair for grazes, abrasions, small cuts, superficial wounds and burns.

The impregnation solution does not contain parabens or other preservatives.

Composition

Impregnation solution: Hyaluronic acid sodium salt 0.3% (main component), glycerol, propanediol, phenethyl alcohol, ethylhexylglycerin, lactic acid, purified water.
Support: Polyurethane, polystyrene, polyethylene, viscose, polyester, polypropylene, polyacrylic adhesive.

Indications

DERMATOP PATCH HI TECH is indicated for the local treatment of grazes, abrasions, small cuts, superficial wounds and burns.

How to use

Clean and dry the affected area thoroughly before application; disinfect if necessary.

1. Grasping the two flaps of the aluminium pack containing the adhesive dressing, pull them to open the pack and remove the dressing.
2. The dressing is double-sided:
 - blue adhesive side, with two overlapping flaps;
 - white outer side.
3. From the blue adhesive side of the dressing, hold the upper wing flap (the longer one) and remove it firmly, exposing part of the adhesive side.
4. Apply the exposed adhesive side onto the wound, making sure that the impregnated pad is centred over the affected area. For optimal adhesion, **carefully press and stretch out** the adhesive part of the dressing to ensure that it is properly stuck on, avoiding creases. When the exposed side is securely stuck on, hold the flap of the blue lower left wing and remove it by gently pulling down, as shown in the figure.
5. Carefully press and stretch out the dressing to ensure proper adhesion, avoiding creases.

6. After adhesion, gently remove the two flaps of the white protective paper on outer side by pulling them along the separation line from the centre toward the edges.

Usage frequency

Apply to the area of wounded skin every 48 hours, until fully healed, for no more than 30 consecutive days.

Contraindications

Individual hypersensitivity to the product components.

Warnings and precautions

For cutaneous use only.

Do not use with products containing quaternary ammonium salts (for example benzalkonium chloride), which may alter the product.

Do not use if package is opened or damaged.

Do not use after the expiry date printed on the package.

The expiry date applies to the product properly stored in its original packaging.

Each adhesive dressing is single-use, i.e. it should only be used once for a single patient and discarded after use.

If it is reused, Fidia farmaceutici cannot be responsible for the performance, functionality and hygiene conditions of the product.

Do not discard the empty packaging into the environment and dispose of according to applicable national guidelines. Avoid contact with eyes. In case of contact with eyes, rinse with water.

Keep out of children's reach and sight.

This product has not been tested in pregnant women,

breast-feeding women or children.

Undesirable effects

No adverse reactions to hyaluronic-acid-based products have been reported.

Storage

Store in the original container at temperature between 2°C and 25°C.

Do not freeze.

Format

REF 03-01A - Pack of five 6x7 cm adhesive dressings for cutaneous use (impregnated pad 3x4 cm)

REF 03-02B - Pack of four 10x10 cm adhesive dressings for cutaneous use (impregnated pad 5x5 cm)

REF 03-02C - Pack of four 8x12 cm adhesive dressings for cutaneous use (impregnated pad 4x8 cm)

REF 03-02D - Pack of four adhesive dressings for cutaneous use: two 6x7 cm, one 10x10 cm, and one 8x12 cm

REF 03-03E - Pack of eight adhesive dressings for cutaneous use: four 6x7 cm, two 10x10 cm, and two 8x12 cm

Instructions for use are included in the pack.

Manufacturer

Fidia farmaceutici S.p.A.
Via Ponte della Fabbrica 3/A
35031 Abano Terme (PD) Italy

Last revision of the package insert: February 2024

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