Sanyrene

Dispositif medical • Medical device • Dispositivo medico

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EN COMPOSITION AND PRESENTATION
SANYRENE is a skin care oil rich in fatty acid glycerides (maize oil: 99%) and containing an anise perfume (1%).

SANYRENE is presented in a tamper-proof spray bottle.

PROPERTIES

- SANYRENE** makes the skin more supple (emollient effect).
- SANYRENE** helps maintain skin oxygenation when the skin is pressed.

INDICATIONS

SANYRENE is intended to be used locally for:

- The prevention of pelvic pressure injuries in patients at risk of pressure injuries,
- The prevention of skin wounds induced by prolonged wearing of protective masks or goggles in health professionals or people in contact with the public.

MODE D'EMPLOI

For external use only on the skin.

For the prevention of pressure injuries

- Wash hands before and after use.
- Make a single spray on the pelvis skin area (trochanter, ischium and/ or sacrum).
- Spread the oil gently with your fingertips for about a minute, without massaging or rubbing in order to avoid any skin damage.

The local use of **SANYRENE** must be carried out within a global management of the patient including, for example, offloading, frequent repositioning and body mobilization, repeated full skin inspections, protection against excessive moisture, incontinence management, and nutrition.

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For the prevention of skin wounds induced by prolonged wearing of protective masks or goggles

- Wash hands before and after use.
- Make a single spray on your fingers.
- Spread the oil gently on the face skin areas (on the nose, on the cheeks, especially at the cheekbones, around the ears, and/ or on the forehead) with your fingertips for about a minute, without massaging or rubbing in order to avoid any skin damage.

Frequency of application

Apply **SANYRENE** two to four times a day, especially when changing position or before wearing a new mask.

Regularity and frequency of application are important to help prevent pressure injuries or skin wounds induced by prolonged wearing of protective masks or goggles.

PRECAUTIONS FOR USE

- It is recommended not to use the same bottle with different patients to avoid microbial contamination between patients.

- Handle the patient carefully.
- The product is oily: wait a few minutes before applying a wound dressing or a bandage on the same skin area, or before wearing a mask or goggles.

WARNINGS

- Keep out of the reach of children.
- Do not spray to or near a flame or incandescent object.
- Do not use if the spray bottle is damaged.
- External use only. Do not swallow. If swallowed,

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rinse mouth immediately, do not induce vomiting. Ingestion may have a laxative effect.

- In case of contact with eyes, mouth or other mucous membranes, rinse immediately with plenty of water.

- Do not massage or rub the skin in order to avoid any skin damage, especially in patients at risk of pressure injuries.

- Talk to your doctor or other healthcare professional if you have a persistent skin redness or unusual appearance of the skin, as it may be a subcutaneous lesion, an allergy, or another skin condition.

- In the case of a pressure ulcer or skin wound, stop applying the product and manage the wound.

ADVERSE SIDE EFFECTS

Allergic contact dermatitis can occur due to the presence of perfume in the product.

Talk to your doctor or other healthcare professional if you get any side effects.

If a serious incident possibly related to the device occurs, please report it to the manufacturer (Laboratoires URGO) and to the national competent authority.

CONTRAINDICATIONS

- Do not use in children.
- Do not apply to a wound.
- Do not use in case of known sensitivity to the product or to an ingredient included in its composition.

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