419265C

UrgoTul Duo

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EN LIPIDO-COLLOID CONTACT LAYER WITH PROTECTIVE AND ABSORBENT PAD

PRESENTATION

Urgotul Duo incorporates Technology Lipido-Colloid (TLC) exclusively developed by Urgo. Urgotul Duo is a non-adhesive, non-occlusive lipido-colloid dressing, composed of a polyester mesh impregnated with hydrocolloid particles (carboxymethylcellulose (CMC)), petroleum jelly and polymers, combined with a protective and absorbent pad (100% viscose).

PROPERTIES:

In contact with wound exudate, the hydrocolloid particles in the Urgotul Duo TLC mass interact with the petroleum jelly component to form a lipido-colloid interface, creating a moist wound healing environment which is favorable to the wound healing process.

The Urgotul Duo pad gives the dressing a protective and absorbent property.

The aerated structure of the mesh and the gas permeability of the pad help prevent maceration.

The low profile and flexibility of Urgotul Duo make it very conformable, suitable for use on wounds located in awkward places.

Due to a chemical composition which offers non-slip properties but not greasy to the touch, Urgotul Duo does not adhere to the wound or the surrounding skin, therefore, making dressing changes painless for the patient and non-traumatic for the wound.

INDICATIONS

Urgotul Duo is indicated for the management of:

- Acute wounds (burns, dermal abrasions, traumatic wounds, post-operative wounds) and chronic wounds (leg ulcers, pressure ulcers and diabetic foot ulcers) at the granulation and epithelialization stage.
- Epidermolysis bullosa (EB).

INSTRUCTIONS FOR USE

Preparation of the wound

- Prepare the wound according to local protocol.

- If an antiseptic is first used, rinse the wound thoroughly with saline solution before applying Urgotul Duo.

Application of the dressing

- The folded wing(s) will enable application to the wound:

- without touching the mesh of Urgotul Duo,

- avoiding dressing adherence to latex surgical gloves.

- Remove the protective transparent film(s).

- Apply Urgotul Duo to the wound with the mesh in contact with it.

- Urgotul Duo can be pre-cut if necessary using sterile scissors, in order to tailor the dressing size to the wound.

- Hold Urgotul Duo in place with a conforming bandage, adhesive tape or an elasticated tubular bandage.

- Apply a compression bandage when prescribed.

Dressing changes

- Urgotul Duo should be changed every 2 to 4 days depending on the type of wound and its clinical condition. At the epithelialization stage, Urgotul Duo may be left in place for up to 7 days dependent on wound condition and protocol (when prescribed with a compression bandage system for venous leg ulcers).

- For patients with epidermolysis bullosa, the dressing should be changed after 1 to 3 days.

WARNINGS AND PRECAUTIONS FOR USE

- Urgotul Duo adheres to latex surgical gloves. It is therefore recommended that gloves be moistened with saline in order to make the handling of Urgotul Duo easier.

- Ensure that the wound is in contact with the mesh and not the pad.

- Do not fold Urgotul Duo.

- If clinical signs of local infection are noted, treatment can be changed to an antibacterial dressing dependent on clinical judgment.

- Urgotul Duo must not be used in a hyperbaric chamber.

- Single use sterile individual packaging: re-using a single use dressing may lead to risks of infection.

- (*) Check that the sterile protector is intact before use. Do not use if package is damaged.

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URGO GmbH; Justus-von-Liebig-Str. 16; 66280 Sulzbach