Pan 540U

Date	25/02/2020		
Client	URGO FRANCE		
Description	IFU URGOSORB Ag		
Dressing size	VARIOUS		
Client Ref	503133		
Finished good	10015710 to 10015723		
No of Colours	1		
Supplier details - TECHNOPRINT			
Item number	10015732 - 21/18/E4		
Size flat	484x160 SIDE 1		
Final Folded Size	80 L x 80mm H $\pm$ 2mm		
View when folded	TITLE OF PRODUCT		
Print process	Offset lithography		
Raw Material ref	RMS006 40-60 gsm		
Version	A1	DBT	

Pan 540U	
Date	25/02/2020
Client	URGO FRANCE
Description	IFU URGOSORB Ag
Dressing size	VARIOUS
Client Ref	503133
Finished good	10015710 to 10015723
No of Colours	1
Supplier details - TECHNOPRINT	
Item number	10015732 - 21/18/E4
Size flat	484x160 SIDE 2
Final Folded Size	80 L x 80mm H $\pm$ 2mm
View when folded	OFFSET OF PRODUCT
Print process	Offset lithography
Raw Material ref	RMS006 40-60 gsm
Version	A1
	DBT

 AMS Studio, Advanced Medical Solutions Ltd.,
Road One, 33 Premier Park, Winsford Industrial Estate,
Winsford, Cheshire CW7 3RT. E: ams.studio@amdsolutions.com
T: +44 (0) 1606 54 5520 F: +44 (0) 1606 54 5626

SUPPLIER
Please acknowledge receipt of artwork as this legend will be the final specification. Check details against This legend information should be correct and inline with the artwork. The supplier must check if this design can be printed on the substrate indicated. Before printing make sure that this legend matches the official order as well. Proceed to repro and proof.
Do not print until you receive an official order
Do not destroy any previous plates until AMS confirm it is ok for you to do so.
Any plates destroyed without this confirmation will be replaced at the supplier's cost.

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END SILVER ALGinate DRESSINGS

PRODUCT DESCRIPTION

UrgoSorb Silver is a highly absorbent, sterile, non woven, antimicrobial dressing composed of a high G (glycolonic acid) alginate, carboxymethylcellulose (CMC) and an ionic silver complex (Silver Sodium Hydrogen Phosphate) which releases silver ions in the presence of wound exudate. As wound exudate is absorbed, the alginate forms a gel which is autolytic, debriding, whilst maintaining a moist environment for optimal wound healing. The gel ensures intact moist wound, without damage to healing tissue.

PROPERTIES

The silver ions released in the presence of wound exudate are an effective antimicrobial agent, against a broad spectrum of microorganisms frequently associated with the bacterial colonisation and infection of wounds, for up to fourteen [14] days.

Furthermore, **UrgoSorb Silver** can also reduce the risk of infected wounds as a result of its effective antimicrobial properties.

UrgoSorb Silver is an effective barrier to penetration by microorganisms.

URGOSORB Silver is indicated for moderate to heavily exuding partial to full thickness wounds including Post-operative wounds

- Trauma wounds (dermal lesions, trauma injuries or incisions)

Leg ulcers

- Pressure ulcers
- Diabetic ulcers

Graft and donor sites

- Chilly wounds
- Superficial and partial thickness burns.

URGOSORB Silver contains Alginate, it may assist in supporting the control of minor bleeding in superficial wounds. This product can be used under compression bandages.

URGOSORB Silver may be used in the management of infected wounds or wounds in which there is an increased risk of infection, under the medical supervision of a healthcare professional.

URGOSORB Silver is indicated for external use only.

METHOD OF USE

Site Preparation:

Cleanse when necessary and irrigate the wound site in accordance with standard protocol.

Remove excess solution from surrounding skin.

Dressing Selection:

Select a size of **URGOSORB Silver** that is slightly larger than the wound.

Use (using sterile scissors) or fold the dressing to fit the wound. Discard any remaining dressing material due to the risk of contamination.

Apply to wound bed directly. Loosely fill wound beds, ensuring the dressing does not overlap the wound margins. Cover and secure **UrgoSilver Silver** with a nonadhesive secondary dressing such as a fixation bandage.

Washing Change and Removal:

UrgoSilver Silver can be used for up to 7 days. Dressing change frequency will depend on wound condition and the patient's condition. Initially, it may be necessary to change the dressing every 24 hours.

Reapply UrgoSilver Silver when the secondary dressing has reached its absorbent capacity or whenever good wound care practice dictates that the dressing should be changed.

Gently remove the secondary dressing.

To remove the **UrgoSilver Silver**, saturate the dressing with sterile saline solution prior to removal.

Gently remove the dressing from the wound bed and dispose of according to local procedures and guidelines.

Do not attempt, irrigate the wound site in accordance with standard protocols prior to application of a new dressing.

Infected wounds:

If the wound is infected, appropriate medical treatment should be initiated.

The application of **UrgoSilver Silver** may be continued at the discretion of the attending medical practitioner/clinician.

CONTRA-INDICATIONS

UrgoSilver Silver is not indicated for use on the following:

- Dry or lightly exuding wounds
- Individuals with a known sensitivity to alginate or silver

To control heavy bleeding.

PRECAUTIONS

The dressing may adhere if used on dry or very lightly exuding wounds. If this occurs and the dressing is not easily removed, moisten it with sterile saline solution prior to removal.

The dressing performance may be impaired by excess use of petroleum-based ointments.

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Date	25/02/2020		
Client	URGO FRANCE		
Description	IFU URGOSORB Ag		
Dressing size	VARIOUS		
Client Ref	503133		
Finished good	10015710 to 10015723		
No of Colours	1		
Supplier details - TECHNOPRINT			
Item number	10015732 - 21/18/E4		
Size flat	484x160 SIDE 2		
Final Folded Size	80 L x 80mm H $\pm$ 2mm		
View when folded	TITLE OF PRODUCT		
Print process	Offset lithography		
Raw Material ref	RMS006 40-60 gsm		
Version	A1		DBT

Avoid contact with electrodes or conductive gels during electronic measurements, e.g. electrocardiograms (ECG) and electroencephalograms (EEG). The dressing must be removed prior to patients undergoing Magnetic Resonance Imaging (MRI) examinations.

In the event of clinical infection, topical silver does not replace the need for systemic therapy or other adequate infection treatment.

Clinicians / Healthcare Professionals should be aware that there are very limited data on prolonged and repeated use of silver containing dressings, particularly in children and neonates.

Urgosorb Silver is intended for short-term use, up to 30 days.

Single use sterile individual packaging: re-using a single use dressing may lead to risks of infection.

Do not use if primary package is damaged/opened.

Do not re-stenilize

Date	25/
------	-----