BORT StabiloPro Knieorthese

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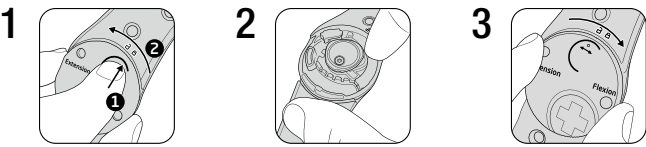
REF 182 150

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D182150|2023-07|007 ML | Rev. 01

DE BORT StabiloPro Knieorthese

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EN BORT StabiloPro Knee Brace

Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

Intended purpose

This medical device is a brace for supporting the physiological guidance of the knee joint with circumferential Velcro fastener restraints and side guidance splints with joints which are adjustable in their scope of movement.

Indications

Mild to moderate side ligament instability of the knee joint, meniscus injuries, gonarthrosis.

Contraindications

Thrombosis risk, extreme varicosis, peripheral arterial occlusive disease (PAOD), lymph drainage disorders, also unclear soft tissue swellings distal to the aid positioned, sensory and circulatory disorders of the region of the body treated, skin diseases in the part of the body treated, latex allergy.

Application risks/Important notes

This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.

- remove the medical device prior to radiological examinations
- remove it during longer periods of rest, e.g. when sleeping
- in case of numbness, loosen the medical device and remove it if necessary
- in case of persistent complaints, consult the physician or specialist retailer
- use the medical device in accordance with therapeutic needs
- only use other products simultaneously after consultation with your physician
- do not make any changes to the product
- do not wear it on open wounds
- do not use in case of intolerance of one of the materials used
- no re-use. This medical aid is intended for treating one patient only
- whilst wearing the brace, please neither use any creams nor ointments on or around it as they can destroy the material

Fitting instructions

The brace can be worn directly on the skin or over the clothes. Pay attention to avoid creasing the clothes in the vicinity of the brace as this can result in pressure sores.

Important note: The brace may only be adapted by expert staff.

Before putting the brace on, the side joint splints must be adapted to the contour of the leg by an expert. For forming, remove the splints from the side splint bags and align to the lateral course of the patient's leg by bending it up (if applicable using a bending iron). Finally, position the joints back in the side splint bags.

For fitting, first open the Velcro straps. Fit the brace so that the patella recess

is directly around the kneecap. Finally, tighten both Velcro fasteners so that the brace supports and compresses but doesn't cut.

Setting the extension/flexion limit: If required, it can be necessary to restrict the scope of the brace's movement. The extension is factory set to 0°, flexion is unrestricted.

1 Remove the outer joint cover. For this, press the crescent-shaped joint cover flap downwards and turn the cover anticlockwise simultaneously.

2 The joints are marked with "A" and "B". Remove the existing limit stop from the joint and take the required limit stop from the limit stop sets provided. They are also marked with the letters "A" and "B", the angular degrees noted on the limit stop wedges. Extension wedges are marked with an "E", flexion wedges with an "F". Insert the limit stop into the joint.

3 Finally, re-attach the joint cover. For this, place the cover on the joint so that the marking on the cover is identical to the open lock symbol. To lock it, turn the cover clockwise under slight pressure until you hear a clicking sound. The following restrictions are available:

- Adjustable extension: 0°, 10°, 15°, 20°
- Adjustable flexion: 0°, 20°, 30°, 45°, 60°, 75°, 90°

Removal

To remove the brace, open the Velcro straps, grasp the lower edge of the brace at the side on the splints and pull it downwards over the foot. Close all the Velcro fasteners in order to avoid damaging other items, e.g. to further clothing.

Material composition

Polyamide (PA), polyurethane (PUR), elastane (EL), viscose (CV), cotton (CO) The sewn in textile label provides the precise material composition.

The product contains latex and can trigger allergic reactions.

Cleaning information

Before washing, remove the joint splints and close the Velcro fastener in order to avoid damaging other items of clothing.

Delicate wash 30 °C Do not bleach Do not dry in a tumble dryer Do not iron Do not clean chemically Do not use fabric conditioner. Stretch back into shape and dry in the fresh air.

Guarantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

Useful life/Lifetime of the product

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

Duty of notification

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA). You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: www.bort.com/md-eu-contact.

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Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: www.bort.com/conformity

Status: 07.2023

MD Medical device | Single patient – multiple use

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