AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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en english

These instructions for use can also be downloaded from:
www.bauerfeind.com/downloads

Dear Customer,

thank you for choosing a Bauerfeind product. If there is a CE mark on the label sewn into your support, this means it is a standard product. If "custom-made solution" is printed on the sewn-in label and there is no CE mark, this means it is a made-to-measure product specially manufactured in accordance with Section 3 No. 8 of the German Medical Devices Act. Please read the **instructions for use** carefully and contact your physician or medical retailer if you have any questions.

Purpose

EpiTrain is particularly effective during movement and exercise treatment. Intermittent compression on the soft tissue parts of the joint is achieved by the knitted fabric and the integral pads made from a viscoelastic material. The improved flow of blood through the joint capsule and the promotion of lymph drainage lead to reduced swelling, pain relief, and improved function. A strap is attached to increase the pain-relieving effect. This can be tightened or loosened or completely removed as needed.

EpiTrain must be fitted for initial use by a trained specialist, otherwise we cannot accept any liability for the product.

Indications

- Irritated tendon – e.g., tennis elbow, golfer's elbow (epicondylopathies)
- Worn / inflamed joint (arthrosis / arthritis)
- Post-operative and post-traumatic irritation
- Muscle and tendon disorders (tendomyopathies)
- Prevention / prophylaxis against relapses

⚠ Risks of using this product

- The use and duration of use must be discussed with the physician who prescribed the EpiTrain. They will also inform you whether the EpiTrain can be used simultaneously with other products, and if so, which ones. Please note all the information you received with the EpiTrain as well as the instructions for use.
- The product may be damaged if it is not used correctly or is altered arbitrarily and the warranty will not be valid.
- The benefits of this support are felt primarily during physical activity. Take the support off during extended rest phases – e.g. sleeping and long periods of sitting down.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves.
- If you experience any unusual changes connected to the use of the EpiTrain – e.g. an increase in discomfort – stop using it and contact a physician immediately.
- If fastened too tightly, the optional strap can lead to bonding of the fasciae in cases of acute inflammatory changes and therefore restriction of movement and pain.

 Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contraindications

Side effects involving the body as a whole have not been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the arms.
3. Impaired lymph drainage – including soft tissue swelling of uncertain origin, located remotely from the fitted aid.

Application instructions

Putting on the EpiTrain

To make the support easy to put on, first undo the strap and loosely attach it to the velour (1 / If using the strap). Fold the top part of the support outward so that the cut outs of the internal massage pads become visible (2). Then take hold of the support near the pad pocket and pull it over the elbow. Make sure that the outer part of the elbow (epicondyle) is positioned in the cut out of the pad, just the same as the inner part of the elbow (3). Then fold the upper edge of the support back into the original position. To increase the pain-relieving effect, clench your hand into a fist, tighten the strap moderately, and refasten the strap without any tension. After this, unclench your fist (4 / If using the strap). As soon as the muscles are tense, a comfortable, relieving compression should be

exerted where the muscle is attached. The strap can also be tightened as required, ensuring that it does not dig into the arm. The strap should be loosened or removed immediately in the event of tingling, blood congestion, swelling, or numbness.

Taking off the EpiTrain

To make the support easy to take off, undo the strap and loosely attach it to the velour. Then take hold of the support at the bottom edge near the pad pocket and pull it toward your hand. Please note: Only attach the strap to the velour surface provided and not to the knitted fabric as the surface of the knitted fabric may be damaged.

Product care instructions

Note: Never expose the support to direct heat (e.g. heaters, sunlight, leaving in your car, etc.) This can impair the effectiveness of the support. Please refer to the care instructions on the sewn-in label. This can be found on the inner part of the knitted fabric.

Please note the following:

1. Fasten the strap.
 2. We recommend using a lingerie bag.
 3. The support is not suitable for tumble drying.
- Wash your support regularly to retain the compression of the knitted fabric.

To make a complaint, please contact the retailer that provided you with the product.

Note: Only clean goods are handled.

Notes on reuse

This product is intended for your treatment only. Do not pass the support on to other people.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. It is your first local point of contact before the manufacturer. Please use the product in line with expert instructions and the enclosed instructions for use. Take note of the care instructions and do not make any alterations to the product. Only aids that function perfectly will help you. Failure to follow the instructions may also compromise the warranty.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

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