53.xx 53.xx.D 53.xx.DD

Latex-Urinal-Kondom/Preventox	Deutsch	2/54
Latex Urinary Sheath/Preventox	English	4/54
Étui pénien en latex/Preventox	Français	6/55
Condom urinario in lattice/Preventox	Italiano	8/55
Latex externe katheters/Preventox	Nederlands	10/56
Urindroppssamlare i latex/Preventox	Svenska	12/56
Latexurinalkondom/Preventox	Dansk	14/57
Lateksinen urinaalikondomi/Preventox	Suomi	16/57
Lateksuridom/Preventox	Norsk	18/58
Lateksist uriinikogur/Preventox	Ísland	20/58
Lateksist uriinikogur/Preventox	Eesti	22/59
Colector de orina peneano de látex/Preventox	Español	24/59
Preservativo urinário em látex/Preventox	Português	26/60
Περιττεικός καθετήρας Λάτεξ / Preventox	Ελληνική	28/60
Latex vizeletkondom/Preventox	Magyar	30/61
Lateksowy cewnik zewnętrzny/Preventox	Polski	32/61
Latexový urinální kondom/Preventox	Čeština	34/62
Prezervativ urinar cu latex/Preventox	Română	36/62
Латексов уриналенкондом/Preventox	Български	38/63
Kondomi Latex-Urinal-Kondom/Preventox	Slovenščina	40/63
Urinarni kondom s lateksom/Preventox	Hrvatski	42/64
Urinarni lateks kondom/Preventox	Srpski	44/64
Urinarni kondom s lateksom/Preventox	Bosanski	46/65
Латекс уринарелн кондом/Preventox	Македонски	48/65

66/49

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Latex Urinary Sheath – Original, Extra Thin**For Single Use****Contents**

- **30 latex urinary sheaths**, for separate fixation, made of natural latex, treated by centrifugation and thus virtually free of chemical residues, with connecting tube with a nipple, connects to all leg bags and bed bags with and without adapter

Order no.	Length of sheath (mm)	Thickness of ridged zone	Thickness of tip
53.xx – Original	165	60 µm	140 µm
53.xx.D – Extra Thin	200	65 µm	110 µm
53.xx.DD	200	65 µm	65 µm

- 11 sizes: Ø 18, 20, 22, 24, 26, 28, 30, 32, 35, 37, 40 mm
("xx" in the order no. represents the size of the sheath (e.g., 53.30 = 30 mm))
- **1 protective cloth**, matching the size of the sheath

Intended use

Urinary sheaths are used for external urinary drainage into a urine collection bag in men with urinary incontinence or if external urinary drainage is desired for other reasons. They are used on intact skin.

Indications

Urinary incontinence of varying causes without residual urine (e.g., neurogenic urinary dysfunctions, hyperactive bladder, urinary incontinence after surgery); urinary drainage in people who cannot manage to go to the toilet by themselves.

Contraindications

Allergies/incompatibilities to latex or other materials in the sheath or adhesive/adhesive tape used. Chronic urinary retention with high intravesical pressure; sphincter dyssynergia with significant urinary retention; constrictions in the urethra (e.g., prostatic hyperplasia) resulting in high residual amounts of urine; skin abrasions on the penis (e.g., wounds, eczema, pressure sores); acute skin diseases (e.g., mycosis, bacterial skin infections); pronounced phimosis.

Important information on hygiene and period of wear

When using continence products, the hygiene guidelines of the national health authorities must be observed. We recommend changing the sheath (the adhesive tape, bag) after 24 hours. The period of wear may also be less than 24 hours depending on the skin type.

Selecting the right size/product

Latex urinary sheaths are available in various sizes. It is essential to select the right size so that the sheath fits properly and does not cause leaks or pressure sores. To select the size, the circumference of the penis must be measured around the middle of the shaft. If it is adhered during erection (see application), the circumference is also measured on the erect penis. The product-specific measuring tapes state the diameter based on the circumference measurement. If leaks or pressure sores occur, the sheath size must be checked and adjusted if necessary.

Risks and warnings

Ask your doctor if latex urinary sheaths are suitable for your incontinence care. Instructions on how to use urinary sheaths should be provided by a qualified nurse.

The use of urinary sheaths may cause skin irritation and skin changes. When removing a urinary sheath, inspect the skin. If you notice any abnormalities, stop using the product and seek medical advice.

If urinary sheaths are worn too tightly, this may cause pressure sores and, in very rare cases (particularly in the case of lack of sensitivity), constrictions and necrosis. In this case, seek a doctor without delay.

When using adhesive tapes, the risk of constrictions/necrosis is greater than when using self-adhesive sheaths or adhesive sprays. When applying the adhesive tape, the application instructions must therefore be followed exactly.

APPLICATION**Preparations**

Wash the skin in the whole genital area thoroughly and dry well. If used on damp skin or on skin that was pre-treated with oily/greasy products, the adhesive effect is reduced.

An erection is not absolutely necessary for adhesion, but if one can be manipulated, we recommend that you apply the sheath to an erect penis as this gives a more stable surface for adhesion and prevents constriction.

Optional: Pull over the protective cloth so that the pubic hair does not stick or obstruct adhesion.

The urinary sheath can be attached to the penis with medical adhesive tape or skin adhesive (e.g., SAUER skin adhesive) suitable for the skin. Please follow the manufacturer's instructions.

Individually adjusting the buffer zone

The length of the buffer zone, which is used to collect spontaneous urine, can be individually set.

To do this, unroll the sheath to the desired length of the buffer zone before fixing it to the shaft.

Fixing with SAUER skin adhesive

Unroll the buffer zone as far as desired and position the sheath on the foreskin without smoothing the foreskin back. If there is no foreskin, place the urinary sheaths directly on the glans (Fig. 1).


With the other hand, apply a ring of adhesive to the penis, using as little as possible but as much as necessary, applying it from the middle of the shaft to the base of the penis. Spread the adhesive. An adhesive area measuring 2–3 cm in width is sufficient. There must be a gap-free adhesive film all around (Fig. 2) Allow the adhesive to dry briefly. If too much solvent remains


Apply the sheath


Apply the adhesive/spray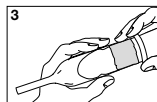

Roll the sheath over the adhesive area

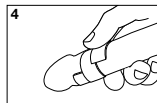Apply the adhesive tape spirally so that it overlaps

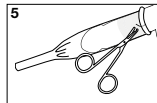Remove the ring

in the adhesive, holes/tears may develop in the condom.

Use one hand to unroll the sheath over the adhesive on the shaft, while using the other hand to keep the penis taut (Fig. 3). Press the sheath carefully around the penis and then rub out longitudinal folds and air bubbles.

Fixing with adhesive tape

Apply the adhesive tape in a spiral so that it overlaps and is not too tight on the penis (Fig. 4). Do not stretch the tape. Then, as described above, unroll the buffer zone as far as desired and position the sheath on the foreskin without smoothing the foreskin back. Slightly stretch the connecting tube. Use one hand to unroll the sheath over the adhesive band on the shaft, while using the other hand to keep the penis taut. Press the sheath carefully around the penis.

Removing the ring

The remaining ring beyond the adhesive area can be cut open or cut/torn off completely using scissors with rounded tips if you are worried about pressure sores or if the penis retracts considerably, e.g., when urinating (Fig. 5). Do not use scissors with pointed tips as they may cause injury. The incision should be circular in shape so that the sheath does not tear. When cutting or tearing off, press down on the adhesive area of the sheath at the same time.

Connecting to the bag

Push the connecting tube of the

sheath on the adapter of the bag and press all around gently. For bags without adapter, push the connecting tube directly on the inlet tube. Make sure that the buffer zone and inlet tube are not twisted or bent.

Removing the sheath

Remove the sheath after 24 hours at the latest and pull or roll off from the back towards the tip of the penis. Do not pull too strongly so as to avoid skin abrasions. An oil pad can be used to undo the adhesive bond. Carefully remove any adhesive remains.

Disposal

Dispose of the sheath in household waste.

	Catalog number		CE marked medical device
	Store in a dry place		Use by (year-month-day)
	Quantity		Not for reuse
	Keep away from sunlight		Pharmaceutical central no.
	Swiss authorised representative		Drug Tariff
	Batch number		List of products and services
	Observe instructions for use		Manufacturer
			Latex

If serious incidents occur when using Latex urinary sheaths, these must be reported to the manufacturer and competent authority of the country in which the user resides.

MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser: MANFRED SAUER GmbH · Neurott 7-15 · 74931 Lobbach · Germany · +49 6226 960100

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Preventox

Adhesive enhancer and skin protection

Preventox cleans, degreases and forms a breathable elastic film on the skin. These properties help improve the adhesive effect when using urinary sheaths while additionally protecting the skin.

Preventox is available as a disposable wipe, in a plastic bottle with spatula and as a roll-on.

Samples (single wipe) are available free of charge –
Tel. 06226 960 100

Preventox is available in the following formats:

- **Single wipes**
Order no. 50.50

- **Plastic bottle with spatula**
Order no. 50.52

- **As a roll-on** to be applied like a roll-on deodorant that is also suitable for those with limited finger functionality.
Order no. 50.58

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