
Mölnlycke®

# Mepilex® Border Post-Op

WITH SAFETAC® TECHNOLOGY

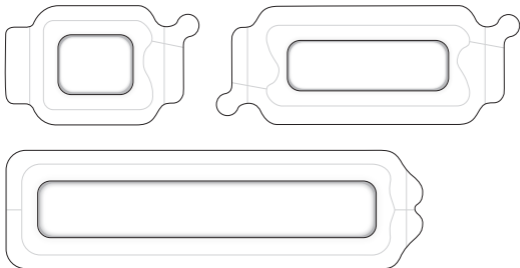STERILE | EO

UK  
CA 0086


CE 2797

Manufacturer/Fabricant


 Mölnlycke Health Care AB  
Entreprenörstråket 21  
SE-431 53 Mölndal, Sweden

[www.molnlycke.com](http://www.molnlycke.com)


For products 6×8, 6×12, 9×10 cm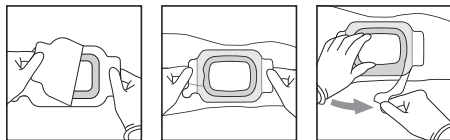

For products 10×15, 10×20, 10×25, 10×30 cm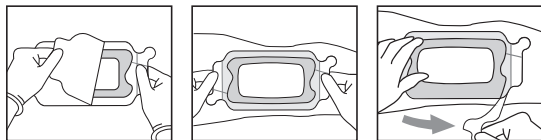

For product 10×35 cm

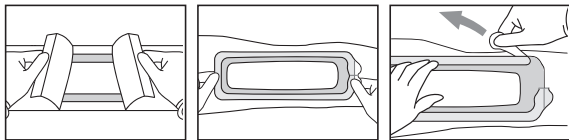[www.molnlycke.com/symbols](http://www.molnlycke.com/symbols)

Single use  
Usage unique

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# Mepilex® Border Post-Op

Self-adherent soft silicone surgical dressing

## Product description

Mepilex Border Post-Op is a self-adherent dressing that absorbs blood exudate and minimises the risk for maceration. The waterproof outer layer protects the wound from dirt and bacteria. The dressing has a Safetac® wound contact layer that is a unique adhesive technology. It minimises pain to patients and trauma to wounds and the surrounding skin at dressing removal.

Mepilex Border Post-Op consists of:

- a transparent outer film which is breathable but waterproof, providing a barrier to external contaminants
- a flexible absorbent pad in two layers: a non-woven spreading layer and a layer with super absorbent fibres
- a wound contact layer consisting of soft silicone adhesive (Safetac) and a film carrier

Dressing material content:

Silicone, polyurethane, viscose, polyester, polyolefins, polyacrylate and superabsorbent acrylate fibers.

## Indications for use

Mepilex Border Post-Op is a self-adhesive absorbent surgical dressing designed for exuding wounds. It is intended for acute wounds, such as surgical wounds, cuts and abrasions.

## Precaution(s)

- If you see signs of infection e.g. fever or the wound or surrounding skin becoming red, warm or swollen, consult a health care professional for appropriate treatment.
- Always consult a health care professional before using Mepilex Border Post-Op on Epidermolysis Bullosa patients.
- Do not use Mepilex Border Post-Op on patient and/or user with known hypersensitivity to the ingoing materials/components of the product.

- Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.
- Sterile. Do not use if sterile barrier is damaged or opened prior to use. Do not re-sterilise.

## Instructions for use

Mepilex Border Post-Op can be used by lay persons under supervision of health care professionals.

1. Cleanse the wound and the surrounding skin in accordance with clinical practice.
2. Dry the surrounding skin thoroughly.
3. The Mepilex Border Post-Op wound pad should overlap the wound by at least 1-2 cm in order to protect the surrounding skin from maceration and fixate the dressing securely.
4. For dressings with one release film, remove the release film partly or completely. For dressings with two release films, remove the small release film partly or completely and then remove the large release film, partly or completely.
5. Apply the adherent side to the wound, allowing initial fixation of the dressing. If applicable, gently remove the remainder of the release film(s) while smoothing the dressing with the other hand to avoid wrinkles. Do not stretch the dressing during application.
6. Grip the application frame and tear the perforation apart.
7. Gently remove the application frame. Avoid wrinkles while fixating the edges by smoothing the dressing with the other hand.

Mepilex Border Post-Op may be left in place for up to 14 days, depending on the condition of the wound and the surrounding skin, or as indicated by clinical practice. Disposal should be handled according to local environmental procedures.

## Other information

If any serious incident has occurred in relation to the use of Mepilex Border Post-Op it should be reported to Mölnlycke Health Care and to your local competent authority.

Mepilex® and Safetac® technology are registered trademarks of Mölnlycke Health Care AB.

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