
Mölnlycke®

Mesalt®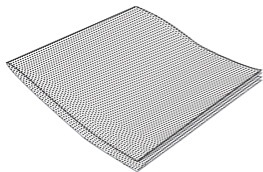

CE 2797

Manufacturer / Fabricant


Mölnlycke Health Care AB
Entreprenörstråket 21


SE-431 53 Mölndal, Sweden

www.molnlycke.com



Mesalt® 5 x 5cm/2 x 2in • 7.5 x 7.5cm/3 x 3in • 10 x 10cm/4 x 4in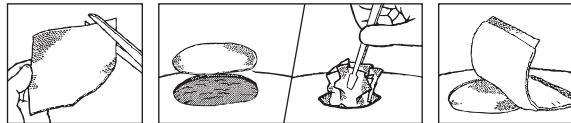

Mesalt® 2 x 100cm / 0.8in x 1.1yd

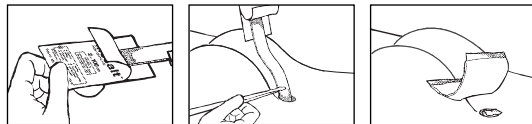www.molnlycke.com/symbols

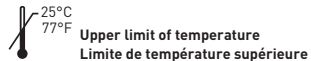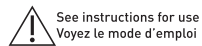Toll free number :

USA 1-800-882-4582

Canada 1-800-494-5134

Australian sponsor address:

Mölnlycke Health Care Pty. Ltd.

12 Narabang Way, Belrose, NSW 2085 AUSTRALIA

CH REP

Mölnlycke Health Care AG

Brandstrasse 24, 8952 Schlieren

- en** Single sterile barrier system
- de** Einzelnes steriles Barriersystem
- fr** Système de barrière stérile unique
- es** Sistema de barrera única estéril
- nl** Enkelvoudig steriel barrièresysteem
- sv** Sterilt singelbarriärsystem
- it** Sistema di barriera sterile singola
- fi** Yksinkertainen steriili sulkujärjestelmä
- pt** Sistema único de esterilização de barreira
- da** Sterilt enkeltbarrieresystem
- el** Ένιαίο σύστημα αποστειρωμένων φραγμών
- pl** System pojedynczej bariery sterylnej
- cs** Systém jedné sterilní bariéry
- hu** Egyszeres sterilgát-rendszer
- no** Sterilt enkeltbarrieresystem
- sl** Enojni sterilni pregradni sistem
- bg** Единична стерилна бариерна система
- ro** Sistem cu barieră sterilă simplă
- sk** Systém s jednou sterilnou bariérou
- tr** Tekli steril bariyer sistemi
- lt** Vieno sterilaus barjero sistema
- lv** Viena sterilās barjeras sistēma
- et** Ühekordselt kasutatav steriilne barjäärisüsteem
- ru** Одиночная стерильная барьерная система
- hr** Jednokratni sustav sterilne barijere
- sr** Jednokratni sistem sterilne barijere
- ar** نظام حاجز معقم أحادي

- en** Medical Device
- de** Medizinprodukt
- fr** Dispositif médical
- es** Producto sanitario
- nl** Medisch hulpmiddel
- sv** Medicinteknisk produkt
- it** Dispositivo medico
- fi** Lääkinnällinen laite
- pt** Dispositivo médico
- da** Medicinsk udstyr
- el** ιατροτεχνολογικό προϊόν
- pl** Wyrób medyczny
- cs** Zdravotnický prostředek
- hu** Orvostechnikai eszköz
- no** Medisinsk utstyr
- sl** Medicinski pripomoček
- bg** Медицинско изделие
- ro** Dispozitiv medical
- sk** Zdravotnícka pomôcka
- tr** Tıbbi cihaz
- lt** Medicinos priemonė
- lv** Medicīniska ierīce
- et** Meditsiiniseade
- ru** Медицинское изделие
- hr** Medicinski proizvod
- sr** Medicinsko sredstvo
- ar** جهاز طبي

- 
- en** Humidity limitation
 - de** Grenzwert für Luftfeuchtigkeit
 - fr** Limite d'humidité
 - es** Limitación de humedad
 - nl** Begrenzing van de luchtvochtigheid
 - sv** Fuktighetsbegränsning
 - it** Limite di umidità
 - fi** Kosteusrajoitus
 - pt** Limitação de humidade
 - da** Luftfugtighedsbegrænsning
 - el** Περιορισμός υγρασίας
 - pl** Ograniczenie wilgotności
 - cs** Omezení vlhkosti
 - hu** Páratartalom korlátozása
 - no** Fuktighetsbegrensning
 - sl** Omejitve vlažnosti
 - bg** Ограничение на влажността
 - ro** Limită de umiditate
 - sk** Obmedzenie vlhkosti
 - tr** Nem sınırlandırma
 - lt** Drėgmės ribos
 - lv** Mitruma ierobežojums
 - et** Niiskuspäärang
 - ru** Ограничение по влажности
 - hr** Ograničenje vlage
 - sr** Ograničenje vlažnosti vazduha
 - ar** حدود الرطوبة

- 
- en** Do not use if package is damaged
 - de** Bei beschädigter Verp. verwerfen
 - fr** Ne pas utiliser si l'emballage est endommagé
 - es** No utilizar si el envase está dañado
 - nl** Niet gebruiken indien verpakking beschadigd is
 - sv** Skall inte användas om förpackningen är skadad
 - it** Non utilizzare se la confezione è danneggiata
 - fi** Älä käytä jos pakkaus vaurioitunut
 - pt** Não usar se embalagem danificada
 - da** Brug ikke, hvis pakken er beskadiget
 - el** Μην το χρησιμοποιήσετε αν η συσκευασία είναι κατεστραμμένη
 - pl** Nie stosować, jeśli opakowanie jest uszkodzone
 - cs** Nepoužívejte, je-li obal poškozen
 - hu** Ne használja, ha a csomagolás sérült
 - no** Må ikke brukes hvis forpakningen er skadet
 - sl** Ne uporabite, če je ovoj poškodovan
 - bg** Не използвайте, ако опаковката е увредена
 - ro** Nu utilizați dacă ambalajul e deteriorat
 - sk** Nepoužívajte, ak je obal poškodený
 - tr** Ambalaj hasarlıysa kullanmayın
 - lt** Nenaudoti, jei pakuotė pažeista
 - lv** Nelietot, ja iepakojums ir bojāts
 - et** Kahjustatud pakendi korral mitte kasutada
 - ru** Не польз., если упак. поврежд.
 - hr** Ne koristiti ako je pakiranje oštećeno
 - sr** Ne koristiti ako je omot oštećen
 - ar** لا يجوز استخدام الجهاز إذا كانت العبوة تالفة

Product description

Mesalt consists of a soft viscose/polyester nonwoven material impregnated with 20% w/w sodium chloride. Mesalt absorbs wound exudate and the sodium chloride is released. Mesalt becomes hypertonic, thereby exudate, bacteria and necrotic material is drawn into the dressing, facilitating the natural wound healing process.

Product material content: viscose/polyester, 20% w/w sodium chloride.

Indications for use

Management of moderate to heavily exuding non-infected/infected wounds such as pressure ulcers, foot ulcers and deep wound management e.g. cavity wounds and fistulas.

Precaution(s)

- Do not use Mesalt on patient and/or user with known hypersensitivity to the materials/components of the product.
- Mesalt should not be used on dry wounds.
- Mesalt should not be allowed to come into direct contact with exposed bone or tendon.
- Infected wounds should be evaluated on a regular basis and treated appropriately.
- Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.
- Sterile. Do not use if sterile barrier is damaged or opened prior to use. Do not re-sterilise.

Instructions for use

Mesalt can be used by lay persons under supervision of health care professionals.

Application:

1. Clean the wound area and debride of any excessive necrotic tissue or eschar.
2. Dry the surrounding skin, but do not dry the surface of the wound.
3. Where clinically indicated, a water-resistant cream or ointment can be used on the surrounding skin.

4. Choose the appropriate Mesalt dressing size. For shallow wounds do not allow Mesalt to overlap the wound edges. For deep wounds, loosely fill the wound.
5. Apply Mesalt dry into the wound
6. Depending on the level of exudate, cover with an appropriate secondary dressing.
7. Where clinically indicated, compression therapy may be used in conjunction with Mesalt.

Dressing change and removal

Mesalt should be changed when saturated (at least once a day), or according to good clinical practice. If the condition of the wound deteriorates, discontinue use and consult a healthcare professional.

Special storage conditions and handling conditions

Mesalt should be stored in dry conditions (below 60% RH) and below 25°C (77°F).

Disposal should be handled according to local environmental procedures.

Other information

For EU-customers, further information on the safety and clinical performance of Mesalt please refer to the 'Summary of Safety and Clinical Performance' for Mesalt. The document is published in the European Database on Medical Devices, EUDAMED, <https://ec.europa.eu/tools/eudamed>.

If any serious incident has occurred in relation to the use of Mesalt, you should report to Mölnlycke Health Care and to your local competent authority.

Mesalt is a registered trademark of Mölnlycke Health Care AB.

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