SAUER-Urinbeutel

SAUER-Urinbeutel	Deutsch	2
SAUER urine bag	English	4
Poches urinaires SAUER	Français	6
Sacca per drenaggio urinario SAUER	Italiano	8
Urineopvangzakken SAUER	Nederlands	10
SAUER-urinpåse	Svenska	12
SAUER-urinposer	Dansk	14
SAUER-virtsapussi	Suomi	16
Sauer-urinposer	Norsk	18
SAUER þvagpokar	Ísland	20
SAUER-uriinikotid	Eesti	22
Bolsa colectora de orina SAUER	Español	24
Sacos de SAUER	Português	26
Ουροσυλλέκτης SAUER	Ελληνική	28
SAUER vizeletgyűjtő zsák	Magyar	30
Worek na mocz SAUER	Polski	32
Urínální sáčky SAUER	Čeština	34
Pungă de colectare a urinei SAUER	Română	36
Уринаторни торби SAUER	Български	38
Urinske vrečke SAUER	Slovenščina	40
Urinske vrećice SAUER	Hrvatski	42
SAUER urin kese	Srpski	44
SAUER vrećice za urin	Bosanski	46
SAUER-уринарна кеса	Македонски	46

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Instructions for use – sterile SAUER urine bag

Instructions on how to use sterile urine bags should be provided by a medical expert. Details of the device's trade name and performance characteristics can be found on the label. Functional bags can be used as leg and/or night bags; bendi, leg, sport, discreet and rectangular bags are designed for use as leg bags. We recommend our leg straps (REF 57.44, 57.48, 57.49) (accessories) for attaching the bags to the leg and our retaining straps (REF 57.38 and 57.39) for securing the inlet tubes to reduce the risk of the tube twisting or pulling on the drainage system.

Intended use and indications

Sterile urine bags serve to collect the urine of persons with an invasive urine drainage system (e.g. transurethral indwelling catheter, suprapubic catheter, etc.).

Risks and warnings

The device must a) not be reused and b) not be used if the sterile barrier is damaged in order not to increase the risk of infection.

Urine bags must not be attached to the leg in the case of peripheral circulatory disorders, diabetic foot syndrome, open leg etc., as there is a risk of infection.

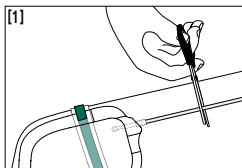
Incorrect use can lead to complications. Urinary retention, e.g. due to a kink in the urine drainage system, can lead to life-threatening urosepsis/autonomic dysreflexia. In this case, contact a doctor immediately; this also applies in the event of infections or persistent urine retention. If serious incidents occur when using "SAUER" urine collection bags, these must be reported to the manufacturer and competent authority of the country in which the user resides.

Precautions during use


- To reduce sources of infection, do not touch the adaptor of the sterile leg/night bag or bring it into contact with surfaces after removing the protective cap. Connect straight away. To cut the inlet tube, use only scissors that have been disinfected beforehand (use a suitable disinfectant, e.g. for surfaces) [1].
- To ensure problem-free urine drainage, attach the leg or night bag below the bladder and avoid kinking or twisting of the inlet tube. With leg bags, straps should be secured under the bag.
- To ensure adequate circulation, do not over-tighten the leg bag straps and check the area under the straps regularly for skin changes, line with a soft pad if necessary.
- Always disconnect the bag before opening the belt.
- Always connect without tension.
- Replace belt if there are signs of wear and tear.

Leg bag instructions for use


The leg bag can be worn on either the left- or right-hand side. The right leg is recommended for leg bags with a white swing tap and the left leg for leg bags with a yellow swing tap in order to avoid accidental opening due to the tap wing resting on the side of the leg. To secure the bag to the leg, thread a leg strap provided for this purpose through two eyelets on the bag. Fasten the strap lying flat between the bag and leg. Depending on the bag volume, use a second strap [4, 5]. To connect the leg bag to a catheter, remove the protective cap of the inlet tube adaptor and push it into the funnel of the urine drain without touching it [2]. Check that it is securely in place [3] [4] [5]. If necessary, secure the inlet tube using a retaining strap [6].

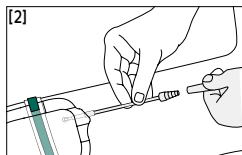
When using the included stepped adaptor, cut the inlet tube to the desired length (straight cut). Attach the adaptor.

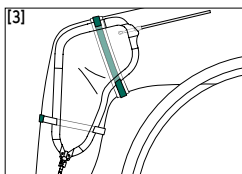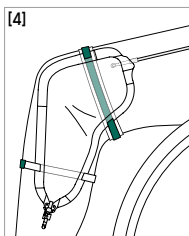
Remove the protective cap from the adapter and connect without touching.


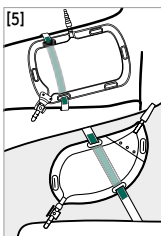
Bendi bag/Bendi bag with needle-free sample port: Multiple eyelets allow the bag to be attached at the knee.

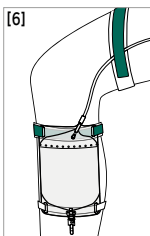
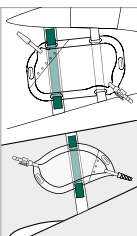
Sport/Discreet bag: Attachment to the thigh (inside or outside) or inside the trouser leg without fixation.

Leg/rectangular/functional bag: Attachment to the thigh or lower leg.

Night bag instructions for use

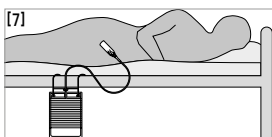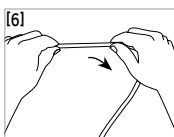After removing from the packaging, roll out the inlet tube as shown to smooth it down [6]. Then hang the bag on the bed frame or on a separate

stand using a bed holder [7]. Finally, remove the protective cap from the adaptor and push this into the funnel of the indwelling urine drainage system without touching it.

Removal and changing

To empty the bag, open the outlet tap and drain the urine into a toilet or other suitable container. Then close the tap again. To change a bag, carefully disconnect from the indwelling urine drainage system without touching the open ends of the connections.

Disposal

Empty before disposal. Dispose of the urine bag in the residual waste.

	Catalog number		CE marked medical device
	Store in a dry place		Use by (year-month-day)
	Do not resterilise		Not for reuse
	Quantity		Pharmaceutical central no.
	Keep away from sunlight		Drug Tariff
	Swiss authorised representative		List of products and services
	Sterilised with ethylene oxide		Do not use if package is damaged
	Single sterile barrier system		Date of manufacture
	Batch number		Manufacturer
	Observe instructions for use		Importer

MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser: MANFRED SAUER GmbH · Neurott 7-15 · 74931 Lobbach · Germany · +49 6226 960100

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MANFRED SAUER GmbH • Neutrott 7-15 • 74931 Lobbach • Germany • +49 6226 960100