AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Purpose

SecuTec Dorso is a medical product. An orthosis¹ for lordosis reduction and relief of the lumbar spine with bridging frames and corset stays.

 **SecuTec Dorso must be individually adapted to the patient by a trained orthotist² in order to gain the maximum effect and optimal wear comfort from this orthosis. If the product is not used properly any claim for warranty will expire.**

Indications

- Spondylolisthesis (with relapsing lumbar sciatica), grade I - III
- Condition after prolapse (e.g. post-operative)
- Condition after discectomy
- Vertebral fractures (lumbar spine) with vertebral body damage
- Osteoporosis
- Moderate to severe lumbar sciatica with muscular deficiencies in cases of significant intervertebral disk protrusions / prolapse
- Severe (pseudo)radicular lumbar syndrome
- Moderate to severe lumbar deformity in cases of facet syndrome / osteoarthritis
- Lumbar spinal canal stenosis

Risks of using this product

 **Important information**

- SecuTec Dorso is a prescription product that should be applied under a medical provider's guidance. SecuTec Dorso should only be worn in accordance with these instructions for use and for the listed areas of application.
- The product should be applied over a T-shirt or an undershirt.
- Incorrect use of the support will void any product liability.
- No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect product performance, thereby voiding any product liability.
- Should you notice increased symptoms or any unusual changes, please stop using the product and please contact your physician without delay.
- Caution: When further loosening the straps or tensioning and fastening systems, or when removing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
- Please consult your medical advisor before using SecuTec Dorso in combination with other products.
- Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids.
- Side effects, that affect the entire organism, have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body may, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves. In such cases, please loosen the straps slightly and have the correct fit and size of the orthosis checked if necessary.

- If you use your product due to a medical prescription or acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product. Individual restrictions may exist which the specialist² and also these instructions for use can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

- Adjust your behavior to the situation associated with your specific handicap / treatment need and any resulting restriction of movement, and avoid anything that could delay recovery where possible. Proceed with particular caution.
- If your perceptive faculties are restricted, if you take painkillers or blood-thinning medication or suffer from dementia, our products must only be worn when prescribed by a physician and under constant supervision of trained specialists².
- If you notice any negative changes or an increase in symptoms while using the product, stop any further use and contact a physician.
- Furthermore, proceed with extreme caution, as your ability to stand and walk (for hand and arm products your ability to grip) can also be heavily impacted.
- For products that affect the locomotor apparatus, check the specific ground characteristics and ensure the ground is not slippery whenever walking, standing or otherwise stressing the part of the body that is receiving treatment. If the ground is smooth, there is an acute risk of slipping and thus injury!

 Contains natural rubber latex (elastodiene), which can cause skin irritation in persons with an allergy. Take off the orthosis if you notice any allergic reaction.

Contra indications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Impaired sensation.
- Severe restriction of the performance of heart and lungs (risk of a rise in blood pressure if the aid is applied during severe physical exertion).

Application instructions

Information for specialists:

If necessary, the corset stays may be adapted.

Fitting and removing SecuTec Dorso

Fitting:

- Place the orthosis around the trunk so that the stays interlinked in the grid are positioned across the back, as shown in **1** (sewn-in labels at the top).
- Place the fingers in the specially provided finger pockets on the two broad, light-gray fastener halves and pull forward uniformly.
- Press the left half of the fastener against the abdomen and pull the right half over it to secure the Velcro fastening **2**.
- When securing the Velcro fastening, remove the fingers from the left finger pocket first, followed by the right **3**.
- Take the two lower LPT (Low Power Tension) straps in both hands and pull forward at the same time to produce the desired level of force and secure both LPT straps against the large Velcro fastening **4**. Repeat this process for the upper LPT tension straps **4** until the orthosis is correctly fastened **5**.
- Correct fitting should be practiced with the patient.

Removal:

- Unfasten the LPT tension straps (first the upper followed by the lower straps) and then refasten without any tension.
- Open the large, light-gray fastener.
- Remove the orthosis and refasten the light-gray fastener.
- For easier handling of SecuTec Dorso, always fasten any opened fasteners directly on the product.

Cleaning instructions

- We recommend washing SecuTec Dorso separately by hand with a mild detergent at 30 °C (86 °F). To avoid damage and to preserve their long-term adhesive strength, we recommend closing the Velcro fastenings before washing. Do not dry in direct heat (e.g. heater, sunlight, etc.).
- Removal of the corset stays is not necessary.
- If the stays are removed, turn back the fasteners at the top of the channels for the corset stays and pull out the stays.
- When reinserting the corset stays after washing, ensure that the stays are inserted in the correct pockets. This can readily be determined by pocket size. Ensure that the corset stays are completely covered by the receiving channels and that the channels are refastened.
- During insertion of the stays, the marking must be at the top and face outwards (away from the body).
- Do not apply excessive force to the stays as this can alter the anatomical contouring of the stays.

This product has been examined by our own quality control system. However, should you have any complaint, please contact your medical retailer or our customer service department.

Part of the body this product is used for

In accordance with the indications.
See Purpose.

Maintenance instructions

With correct handling and care, the product is practically maintenance-free.

Assembly and fitting instructions

The SecuTec Dorso is supplied in a standard size and ready for use. The bridging frame must be individually adapted by a trained specialist and positioned in the knitted section.

Technical specifications / parameters, accessories

The SecuTec Dorso is a orthosis for the back. It consists of a knitted section, a velour pocket for inserting the bridging frame, an abdominal pad and four double-sided micro Velcro fastenings. The velour pocket and bridging frame are available as spare parts on request.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disclaimer

For use of the SecuTec Dorso, the following applies:

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our product, please seek the advice of this medical specialist, as this is the only way to assess the effect of our SecuTec Dorso on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of the medical specialist as well as all instructions in this brochure or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

SecuTec Dorso can readily be disposed of with the domestic refuse.

Material content

Polyamide (PA), Spring Steel Strip C75, coated, Polyurethane (PUR), Polyoxymethylene (POM), Elastodiene (ED), Polyethylene (PE), Spring steel wire, galvanized, Cotton (CO), Polyester (PES), Spandex (EL), Copolyester (TPC-ES)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2023-06

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

² A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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