Spinova® Osteo					
Größe Size	S 2	S 3	M 3	M 4	L 4
Rückenlänge Back length (cm)	50–58	50–58	59–67	59–67	68–76
Beckenumfang Pelvic circumference (cm)	80–90	90–100	90–100	100–110	100–110

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Stand der Information: 2022-03

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

Spinova Osteo is a medical product. It is an orthosis¹ for stabilization and active straightening of the spine, with free abdomen and chest area.

Indications

The Spinova Osteo helps with all indications that require active straightening to relieve and correct the lumbar and thoracic spine and any restriction of upper body movement.

- Stable, osteoporotic vertebral body fractures
- Osteoporosis
- Painful hunched back
- Muscular insufficiency

Risks of using this product

⚠ Caution*

Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

Your Spinova spine orthosis must only be used according to the specifications in these instructions for use and for the applications listed (intended use / part of the body this product is used for). It is a prescribed product that should be applied under a physician's guidance.

To ensure the best possible fit and optimal function, the orthosis must be individually adjusted to you by a trained specialist² (adjusting the length of the straps and pads, fitting the reclinator to the shape of your back). Only a trained specialist may carry out the fitting of the orthosis and provide instruction.

If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

No side effects that affect the entire body have been reported to date. This assumes the orthosis is used / fitted correctly. Do not put the Spinova Osteo on too tightly as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur.

If you are wearing the orthosis after surgery, have the wound checked regularly by a physician.

Combination with other medical products should be discussed beforehand with the physician treating you.

Application instructions

Please refer to the care instructions on the sewn-in label. The sewn-in label contains information on the size, manufacturer, care instructions, and CE marking and is located on the inside of the hip support.

The Spinova Osteo must be applied over a T-shirt or undershirt. You can also wear the Spinova Osteo over your outer clothing.

Note: Never expose the product to direct heat (e.g. heaters, sunlight, in your car, etc.)! This may damage the material, which can impair the effectiveness of the orthosis.

Putting on the orthosis

➊+➋ Put the tensioning straps ➌ with the shoulder pads ➍ over both shoulders in the same way as you would put on a backpack. Make sure that the straps are not twisted. First position the hip support ➎ on your pelvis so that the upper edge is roughly level with your iliac crest. ➏ To do this, place your fingers in the hand straps provided on the support's fastening ➐ and ➑ pull the support outward in both directions initially to stretch the material. ➒ Then fold both halves of the fastening round to the front. ➓ Fasten the hip support by positioning the right half of the fastening on top of the left half so that it can be fastened to it. The hip support should fit tightly but comfortably ➔. ➕+➖ Now take hold of both tensioning straps by releasing the blue finger pockets ➗ from the strap attachment point ➘. ➙ Pull both strap ends ➚ forward, alternating from one to the other, until you achieve the required straightening and stabilization effect. ➛+➜ Now secure the blue strap ends ➚ flush on the center of the support's fastening ➐. Check the fit of the support and correct the position of the shoulder pads if necessary ➐. The tops of the pads must not touch the reclinator ➌ under any circumstances ➛, otherwise the straightening function of the orthosis will be reduced. **Have the shoulder pads ➐ shortened by your specialist if necessary.**

Note: If the strap ends overlap at the front when you have put the orthosis on, have the straps shortened by your specialist.

Taking off the orthosis

Release the blue strap ends ➚ from the front of the support's fastening ➐ and secure these to the strap attachment points ➘ at the side again. Then loosen the tensioning straps ➌ and take the orthosis off. Make sure to refasten the support's fastening ➐ after taking it off to avoid damage.

Cleaning instructions

If necessary, you can wash the Spinova Osteo by hand with a mild cleaning detergent. Let the orthosis air dry. **Please do not dismantle the orthosis for cleaning.**

Part of the body this product is used for

In accordance with the indications. See Purpose.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Components

- A – Reclinator pocket with reclinator
- B – Hip support
- C – Support fastening with hand straps
- D – Lateral strap attachment points
- E – Lateral strap crossing points
- F – Tensioning strap system
- G – Shoulder pads
- H – Pocket for reclinator pocket
- I – Dorsal buckles
- J – Strap ends with finger pockets

Assembly and fitting instructions

The Spinova Osteo is supplied as a pre-assembled orthosis in five standard sizes. The reclinator, straps, and shoulder pads must be individually adjusted by a trained specialist².

For an extended sizing system, the Spinova Osteo also allows customized combinations with a choice of six different hip support sizes and three different reclinator lengths.

Technical specifications / parameters

The Spinova Osteo is an osteoporosis orthosis for active relief and correction of the lumbar and thoracic spine using biofeedback (reminder function) and movement restriction in the sagittal plane.

It consists of a padded dorsal splint (reclinator) ➌ fitted with a strap system ➌ running over the shoulders and around the torso. The circular strap is an elastic pelvic support ➎ that has a fastening with large hand straps ➐ and strap attachment points at the sides ➘. The reclinator ➌ is located in the reclinator pocket and linked to the strap system ➌. The tensioning straps have shoulder pads ➐, lateral crossed straps ➓, dorsal buckles ➑, and finger pockets ➗ on the strap ends at the front.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the Spinova Osteo have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Aluminum (AlMgSi1), Polyurethane (PUR), Polyether, Spandex (EL), Polyester (PES), Polyethylene, low-density (PE-LD), Ethylene Vinyl Acetate (EVA), Polyketone (PK), Polypropylene (PP)

MD – Medical Device

UDI – DataMatrix Unique Device Identifier UDI

¹Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso

²A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

*Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product).

Information for specialists²

Individual fitting / assembly

Tools required:

Scissors (not included in the delivery)

Adjusting the reclinator: Spinova Osteo orthoses are available in five ready-made sizes. The shape of the reclinator must be adjusted to the appropriate back shape when the patient is standing ➔.

The reclinator does not necessarily need to be removed from the reclinator pocket for reshaping ➑. If required, bending irons with rounded edges are to be used.

Adjusting the tensioning straps and shoulder pads:

The length of the straps usually need adjusting. Both the front and back strap ends can be adjusted ➙+➜.

The shoulder pads can also be shortened in two places. To do this, cut along the welded seam ➖.

Assembling partially assembled versions: The 18 partially assembled versions of the Spinova Osteo (six different hip supports and three reclinators) must first be put together. To do this, slide the reclinator in the reclinator pocket into the dorsal pocket on the hip support ➖, then secure the dorsal buckles in the eyelets of the reclinator ➑. The dorsal strap ends can be attached at three different heights to optimize the position of the straps ➑. The pocket for the reclinator has relevant markings ➙.

The reclinator shape, strap lengths, and shoulder pad lengths are adjusted as described above in the ready-made versions.

Note: The top of the reclinator must not stick out above the shoulders. Check the optimal fit of the orthosis after adjusting it for the patient. **Without individual fitting, the function of the orthosis may be limited!**

Practice the process of putting on and taking off the orthosis correctly with the patient. Ask the patient if the orthosis is sufficiently stable and comfortable.

No improper modifications may be made to the product. **Failure to comply with this requirement may adversely affect product performance, thereby canceling any product liability.**

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