AUSTRIA
Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

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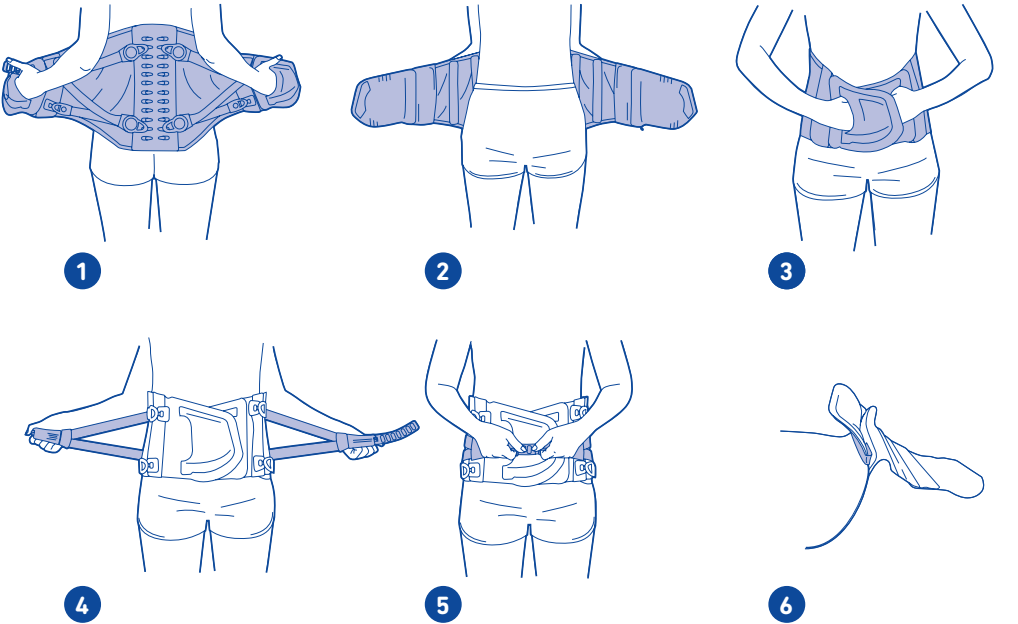
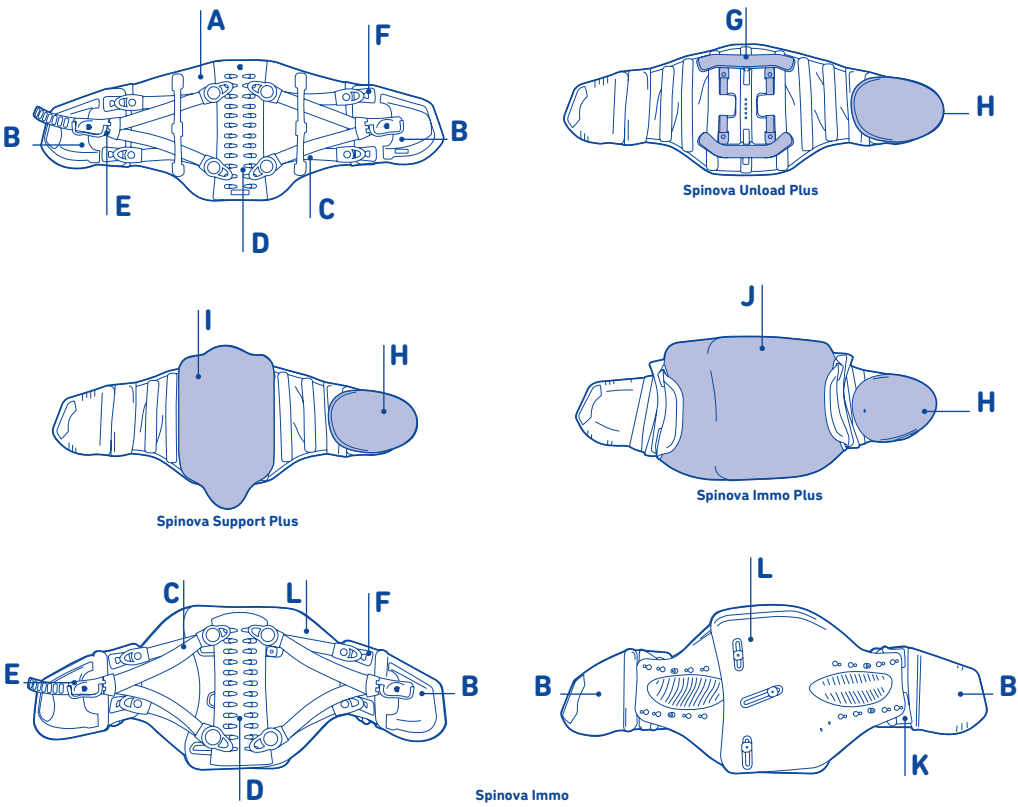
Material	
Base corselet	Spinova Immo
Polyamide (PA)	Polyamide (PA)
Polyurethane (PUR)	Thermoplastic Polyurethane (TPU)
Spandex (EL)	Polyurethane (PUR)
Polyester (PES)	Polyoxymethylene (POM)
Iron with Polyamide	Elastane (EL)
Polyoxymethylene (POM)	Bridging frame – Spinova Unload Plus
Front closure	Polycarbonate (PC)
Thermoplastic polyurethane (TPU)	Aluminium
Polyurethane (PUR)	EthyleneVinylAcetate (EVA)
Spandex (EL)	Polyoxymethylene (POM)
Polyester (PES)	Polyamide (PA)
Polypropylene (PP)	Polyurethane (PUR)
Tensioning strap system	Acrylate (AK)
Styrene-Ethylene (TPS-SEBS)	Back Panel
Polyamide (PA)	Spinova Support Plus
Elastane (EL)	Polyamide (PA)
High-Grade Steel	Glass fiber (GF)
Polyoxymethylene (POM)	Polyester (PES)
Thermoplastic Polyurethane (TPU)	Polyurethane (PUR)
Polyurethane (PUR)	Polypropylene (PP)
Polyester (PES)	Abdominal pad
Polypropylene (PP)	Spinova Unload Plus Spinova Support Plus Spinova Immo Plus
Polyurethane (PUR)	Polyethylene (PE)
Shell system Spinova Immo/Immo Plus	Ethylene Propylene Diene M-Class Rubber (EPDM)
Polyethylene, low density (PE- LD)	Polyurethane (PUR)
High-Grade Steel	Polyamide (PA)
Polyamide (PA)	Cotton (CO)
Polyurethane (PUR)	Polyester (PES)
Shell cover	Elastane (EL)
Polyester (PES)	
Elastane (EL)	
Polyamide (PA)	

de	deutsch	2	hr	hrvatski	54
en	english	6	sr	srpski	57
fr	français	9	sl	slovenski	61
nl	nederlands	13	ro	romanian	64
it	italiano	17	tr	türkçe	68
es	español	21	ru	русский	72
pt	português	25	ee	eesti	76
sv	svenska	29	lv	latviešu	79
no	norsk	32	lt	lietuvių	83
fi	suomi	36	ja	日本語	86
da	dansk	39	ko	한국어	89
pl	polski	43	ar	عربي	93
cs	česky	47	zh	中文	97
hu	magyar	50			

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

The following information relates to these products:

Spinova Unload Plus

Spinova Support Plus

Spinova Immo Plus

Spinova Immo

Please also read the specific instructions relating to your product on the following pages.

Risks of using this product

Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

Your Spinova spine orthosis must only be worn in accordance with the specifications contained in these instructions for use and used for the areas of application listed (intended use / parts of the body this product is used for). It is a prescribed product that should be applied under a physician's guidance. To ensure the best possible fit, the orthosis must be individually adjusted by a trained specialist¹. Please refer to the care instructions on the sewn-in label. The sewn-in label contains information on the size, manufacturer, care instructions, and CE marking and is located on the inside of the base support (in the case of the Spinova Immo, it is located on the inside of the back setting device).

Only a trained specialist may carry out the initial fitting of the orthosis and provide instruction. Use in combination with other products, e.g. as part of compression treatment (compression body support, lymph compression stockings), must be discussed beforehand with your physician. Should you notice any unusual changes or increased symptoms, please contact your physician without delay. No product liability is accepted in the event of improper use. Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids. No side effects that affect the entire body have been reported to date. Correct usage / fitting is assumed. Do not put the orthosis on too tightly as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur. For this reason, check the fit of the product if you feel unpleasant pressure and consult the specialist in this regard or for general questions about handling.

Caution: When loosening the straps or tensioning and fastening systems, or when removing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.

After surgery, make sure that the wound is regularly checked by a physician.

If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product. Restrictions may exist given the specific indication which the specialist retailer can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

Note: Never expose the product to direct heat (e.g. heaters, sunlight, in your car, etc.)! This may cause damage to the material, which can impair the effectiveness of the orthosis.

Application instructions

Putting on the orthosis

1 Position the orthosis centrally, at waist height, so that the lower back is covered. The stamped arrows on the inside of the abdominal fastening should be pointing upward. **2** Slide your fingers from the side into the finger pockets provided on the abdominal fastening and pull the pockets evenly to the side. Now pull the fastenings forward. **3** Lift the abdomen slightly from below with the left half of the fastening and position the right half of the fastening on top of the left half so that it can be fastened to it.

4 Now grasp the ends of the tensioning strap system on both sides. The handles on the belt fastening fold out. Pull the tensioning straps to the sides and away from one another. **5** Bring the belt fastening and eyelet together around the lower part of the abdomen and attach the hook to the eyelet on the fastening. You can attach the hook to different rows of eyelets if required, to vary the compression of the strap system to suit individual needs.

After putting on the orthosis, ensure that there are no creases in the mesh material. If there are visible creases, pull both abdominal fastenings on the base support further over each other before fastening them.

Note: When sitting for long periods of time, it is a good idea to loosen the belt fastening slightly to relieve pressure on the abdomen.

Taking off the orthosis

Undo the belt fastening on the tensioning strap system. Then reach into the pockets of the abdominal fastening and unfasten the closure. After removing the orthosis, refasten the abdominal fastening to avoid damage.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

- Skin disorders / injuries to the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
- severe restriction of the performance of the heart and lungs (risk of a rise in blood pressure if the aid is worn during intense physical exertion)

Cleaning instructions

If necessary, you can wash the orthosis by hand with a mild cleaning agent. Leave the orthosis to air dry.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the product have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

You can dispose of the product in accordance with national legal requirements.

Version: 2023-04

Spinova Unload Plus

Intended use / part of the body this product is used for

Spinova Unload Plus is a medical product. It is an orthosis for reducing and providing lordosis relief of the lumbar spine with removable bridging frame.

Indications

- Vertebral fractures (lumbar spine) with vertebral body damage – relieving the posterior edges of vertebrae
- Spondylolysis / spondylolisthesis, grades I – II
- Most severe lumbar deformity in cases of facet syndrome / osteoarthritis
- Most severe lumbosciatica with muscular deficiencies in cases of significant intervertebral disk protrusions / herniated disks
- Lumbar spinal canal stenosis, also with paresis (non-surgical, post-operative)
- Most severe (pseudo) radicular lumbar syndrome
- Foraminal stenosis

Components

- A – Base support
- B – Abdominal fastening with finger pockets
- C – Tensioning straps / tensioning strap system
- D – Back setting device
- E – Belt fastening
- F – Side clips
- G – Bridging frame
- H – Abdominal pad

Assembly and fitting instructions

The Spinova Unload Plus is supplied as a pre-assembled orthosis in standard sizes. The corset stays, the back setting device, the abdominal fastening, the bridging frame, and the tensioning strap system with a belt fastening must be individually adjusted by a trained specialist. The bridging frame and abdominal pad must be fastened to the base support. See **'Cleaning instructions'** for information on how to remove and attach the frame.

Cleaning instructions

If necessary, you can clean the Spinova Unload Plus by hand with a mild cleaning agent. Unfasten the bridging frame and the abdominal pad before washing **G** **H**. Refasten the bridging frame after drying. To do this, place the orthosis in front of you with the inside facing upward. The arrows on the inside of the abdominal fastenings should be pointing upward. The arrows on the bars of the bridging frame should be pointing upward. Secure both bars on the bridging frame. Push each Velcro strip through the eye of the bar from the front **VII**. Now attach the abdominal pad to the right abdominal fastening **H**.

Technical specifications / parameters

The Spinova Unload Plus is made up of a base support with corset stays, a back setting device, an abdominal fastening, a strap system with a belt fastening, a bridging frame, and an abdominal pad.

Spinova Support Plus

Intended use / part of the body this product is used for

Spinova Support Plus is a medical product. It is an orthosis for stabilizing the lumbar spine with an anatomically contoured back pad.

Indications

- Vertebral fractures (lumbar spine) with vertebral body damage – relieving the anterior edges of vertebrae
- Spondylolysis / spondylolisthesis, grade II
- Moderate lumbosciatica with muscular deficiencies in cases of significant intervertebral disk protrusions / herniated disks
- Lumbar spinal canal stenosis, also with paresis (non-surgical, post-operative)
- Moderate lumbar deformity in cases of facet syndrome / osteoarthritis
- Moderate (pseudo) radicular lumbar syndrome
- Severe degeneration / muscular insufficiency
- Condition after herniated disks (post-operative / non-surgical)
- Condition after discectomy
- Condition after spondylolysis / kyphoplasty (one or more levels)
- Osteoporosis

Components

- A – Base support
- B – Abdominal fastening with finger pockets
- C – Tensioning straps / tensioning strap system
- D – Back setting device
- E – Belt fastening
- F – Side clips
- H – Abdominal pad
- I – Bridging pad

Assembly and fitting

The Spinova Support Plus is supplied as a pre-assembled orthosis in standard sizes. The corset stays, the back setting device, the abdominal fastening, and the tensioning strap system with a belt fastening must be individually adjusted by a trained specialist. The bridging pad and abdominal pad must be fastened to the base support. See **'Cleaning instructions'** for information on how to remove and attach the pads.

Cleaning instructions

If necessary, you can clean the Spinova Support Plus by hand with a mild cleaning agent. Unfasten both pads before washing **I** **H**. Refasten both pads to the Spinova Support Plus after drying. To do this, place the orthosis in front of you with the inside facing upward. The arrows on the inside of the abdominal fastenings should be pointing upward. Fasten the back pad firmly onto the center of the support with the tapered end pointing downward **I**. Now attach the abdominal pad to the right abdominal fastening **H**.

Technical specifications / parameters

The Spinova Support Plus is made up of a base support with corset stays, a back setting device, an abdominal fastening, a tensioning strap system with a belt fastening, a bridging pad, and an abdominal pad.

Spinova Immo Plus

Intended use / part of the body this product is used for

Spinova Immo Plus is a medical product. It is an orthosis for immobilizing the lumbar spine with mobilization function with removable shell. Its special design permits controlled treatment according to an individually adaptable multi-stage regimen:

Stage 1 (Stabilization phase):

To secure the spine after surgery or at the start of non-surgical treatment, use the orthosis in combination with the shell.

Stage 2 (Mobilization phase):

During the initial stages of mobilization with a progressive recovery process, the shell is removed. The base support is reinforced with the corset stays in the affected lumbar / sacral area.

Please ask your physician when you can remove the shell.

Indications

- Vertebral fractures (lumbar spine) with significant vertebral body damage – relieving the anterior and / or posterior edges of vertebrae
- Tumors (metastases)
- Condition after Spondylolysis / kyphoplasty (one or more levels)
- Most severe (pseudo) radicular lumbar syndrome
- Most severe lumbosciatica with muscular deficiencies in cases of significant intervertebral disk protrusions / herniated disks
- Lumbar spinal canal stenosis with paresis (non-surgical, post-operative)
- Foraminal stenosis
- Osteochondrosis
- Severe degeneration / muscular insufficiency
- Severe lumbar deformity in cases of facet syndrome / osteoarthritis
- Condition following herniation (non-surgical / post-operative)
- Condition following discectomy
- Spondylolysis / spondylolisthesis, grades I – III

Components

- A – Base support
- B – Abdominal fastening with finger pockets
- C – Tensioning straps / tensioning strap system
- D – Back setting device
- E – Belt fastening
- F – Side clips
- H – Abdominal pad
- J – Shell with fabric top cover

Assembly and fitting instructions

The Spinova Immo Plus is supplied as a pre-assembled orthosis in standard sizes. The shell, the stays, the abdominal fastening, and the tensioning straps must be individually adjusted by a trained specialist. The abdominal pad must be fastened to the base support.

6 To detach the fabric top cover, slip both the rubber bands on the base support over the front edges of the shell. Then remove the fabric top cover. To detach the shell from the orthosis, undo the middle screw on the back setting device. Remove the shell from the base support, then you can use the base support without it.

Cleaning instructions

The shell's fabric top cover can be washed in a machine at 40 degrees centigrade. Wipe the shell with a soft cloth and a mild cleaning agent. When using the Spinova Immo Plus without the shell (mobilization phase), the orthosis can be cleaned by hand if necessary. Then leave the orthosis to air dry.

Technical specifications / parameters

The Spinova Immo Plus is made up of a base support with corset stays, a back setting device, an abdominal fastening, a tensioning strap system with a belt fastening, a shell, and an abdominal pad.

Spinova Immo

Intended use / part of the body this product is used for

Spinova Immo is a medical product. It is an orthosis for immobilizing the lumbar spine.

Indications

- Immobilization of the lumbar spine (pre-operative, post-operative, post-traumatic, inflammatory diseases)
- Non-surgical treatment of simple and stable fractures

Components

- B – Abdominal fastening with finger pocket
- C – Tensioning straps / tensioning strap system
- D – Back setting device
- E – Belt fastening
- F – Side clips
- K – Shell connector
- L – Shell

Assembly and fitting instructions

The Spinova Immo is supplied as a pre-assembled orthosis in standard sizes. The shell, the stays, the abdominal fastening, and the tensioning straps must be individually adjusted by a trained specialist.

Cleaning instructions

Wipe the shell with a soft cloth and a mild cleaning agent. Then leave the orthosis to air dry.

Technical specifications / parameters

The Spinova Immo consists of a shell system, an abdominal fastening, a shell connector, a back setting device, and a tensioning strap system with a belt fastening.

Specialists¹

The following information relates to these products:

Spinova Unload Plus

Spinova Support Plus

Spinova Immo Plus

Spinova Immo

Please also read the specific instructions relating to each product on the following pages.

General instructions

Spinova products come in preset sizes, but can be individually adjusted if necessary depending on the areas of application (indications) and the patient's body shape (anatomy). Check the optimal and individual fit of the orthosis after the first fitting. Practice the correct procedure for putting on the orthosis with the patient **(1–5)**. No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect product performance, thereby voiding any product liability. The Spinova spine orthosis is intended for the treatment of one patient only.

Make sure that there are no creases in the orthosis (does not apply to the Spinova Immo). If creasing occurs, patients should put the orthosis on again and overlap the two halves of the abdominal fastening more before they are fastened. Another way to reduce creasing is to adapt the abdominal fastenings to the patient's body shape.

Individually adjusting the orthosis

Tools required

- Scissors
- TX 40 Torx key / 0.25 Nm torque wrench recommended (available as an add-on)

Adapting the abdominal fastenings to the patient's body shape

You can adjust the angle and width of the abdominal fastenings to suit individual requirements. To do so, loosen the abdominal fastenings and fasten them in a new position. Any protruding velor fabric on the base support can be cut off with fabric scissors once the product has been fitted. Protruding Velcro must be covered with a velor strip to avoid damage. In principle, you can also cut off the outer Velcro strip completely if it is not required **(1)**.

Adjusting the back setting device

Unhook the blue strap ends from the back setting device. Undo the screw (Spinova Immo / Immo Plus). Remove the back setting device with the integrated aluminum stays and shape it to suit the patient's indication **(1)**.

Adjusting the side clips

To avoid local pressure peaks on the spine or ribs, you can reposition the side clips individually in the velor area **(1)**.

Note: After adjusting the abdominal fastenings and the side clips, you may need to re-adjust the strap system to the abdominal circumference.

Adjusting the abdominal pad (does not apply to the Spinova Immo)

Attach the abdominal pad to the left-hand side of the underside of the left abdominal fastening.

Adjusting the tensioning strap system Adjusting the abdominal circumference

Put the orthosis on the patient as described above. Do not close the belt fastening. Now set the locking devices **(V)** on the four blue ends of the strap system to the open lock symbol **(L)**. The patient should then pull the ends of the tensioning strap system to the sides and away from one another, and then close the belt fastening in a central position on the rows of eyelets. Then set the locking devices on the ends of the straps to 'closed' **(L)**. Pull the strap system taut by tightening all four Torx screws evenly in a clockwise direction. Count the number of turns of the screws. Alternatively, we recommend that you tighten the screws using a Torx torque wrench with a torque of 0.25 Nm (available as an add-on) **(V)**.

Instruct the patient to undo the belt fastening again and pull the tensioning straps to the sides and away from one another, then close the fastening again. It should be possible to close the belt fastening in a central position, so that the patient can continue to use the orthosis even if they gain or lose weight. If it is not possible to close the fastening in a central position, repeat the entire adjustment process.

Ask the patient if the orthosis is sufficiently stable and comfortable.

Adjusting the tensioning strap system in accordance with treatment requirements

The height of the tensioning straps can be adjusted for individual fitting, lordosis, and reducing lordosis. The back setting device has 12 different attachment positions for this purpose **(VI)**.

Note: Always attach the tensioning straps to the back setting device symmetrically.

After changing the position of the tensioning straps, it may be necessary to re-adjust the tensioning strap system to the abdominal circumference.

Spinova Unload Plus

Fitting / assembly

Adjust the abdominal fastening, back setting device, side clips, abdominal pad, and tensioning strap system as described above.

Adjusting the bridging frame **(VI)**

To adjust the bridging frame, unfasten it from the base support. Remove the two Velcro strips from the bridging frame and open the two Velcro tabs on the base support. The aluminum stays on the bridging frame can be individually shaped. Then reattach the bridging frame, positioning it according to the patient's body shape so that its delordosing function works to optimum effect. The arrows on the bars of the frame and the inside of the abdominal fastenings should be pointing upward. Please note that the Spinova Unload Plus is available in two heights.

Spinova Support Plus

Adjust the abdominal fastening, back setting device, side clips, abdominal pad, and tensioning strap system as described above.

Positioning the back pad **(VII)**

Place the orthosis in front of you with the inside pointing upward. The arrows on the inside of the abdominal fastenings should be pointing upward. Fasten the back pad firmly onto the center of the support with the tapered end pointing downward. The back pad should end approximately 3 cm below the shoulder blades. Now attach the abdominal pad to the left abdominal fastening. Please note that the Spinova Support Plus is available in two heights.

Spinova Immo Plus

Fitting / assembly

Adjust the abdominal fastening, back setting device, side clips, abdominal pad, and tensioning strap system as described above.

Tools required

2.5 mm Allen key (available as an add-on)

Adjusting the shell Removing the shell from the orthosis

(6) Undo the middle screw on the back setting device. Slip both the rubber bands on the base support over the front edges of the shell. Remove the base support from the back shell. If necessary, remove the fabric top cover from the shell.

Setting the height, width and angle of the shell

After removing the shell from the base support, undo the nut in the centre of the shell by hand as well as the upper and lower screws, so that both halves of the shell can be moved easily toward or away from each other. Position the shell on the patient's torso so that the recesses of the pelvic frame sit directly above the iliac crests. Now slide the two halves of the shell toward each other until the shell lies flat against the entire torso **(X)**. Then tighten all the screws in the shell. The upper edge should end approximately 3 cm below the shoulder blades. If necessary, the upper edge can be shortened to change the height of the shell. Please note that the Spinova Immo Plus is available in two heights. To attach the shell to the base support, carry out the steps listed in the **Removing the shell** section in reverse.

Note: The plastic can be cut with standard leather shears. Make sure that you smooth off the cut edge.

Spinova Immo

Fitting / assembly

Adjust the abdominal fastening, back setting device, side clips, and tensioning strap system as described above.

Tools required

2.5 mm Allen key (available as an add-on)

Adjusting the shell Removing the shell from the orthosis

Remove the strap system from the back setting device by detaching the blue and grey ends from the side clips. Remove the two Velcro patches from the back of the shell. Undo the middle screw on the back setting device and remove the back setting device from the shell.

Setting the height, width and angle of the shell

After removing the tensioning strap system, undo the nut in the centre of the shell by hand as well as the upper and lower screws, so that both halves of the shell can be moved easily toward or away from each other. Position the shell on the patient's torso so that the recesses of the pelvic frame sit directly above the iliac crests. Now slide the two halves of the shell toward each other until the shell lies flat against the entire torso **(X)**. Then tighten all the screws in the shell. The upper edge should end approximately 3 cm below the shoulder blades. If necessary, the upper edge can be shortened to change the height of the shell. To attach the tensioning strap system to the shell, carry out the steps listed in the **Removing the shell** section in reverse.

With the Spinova Immo, the circumference can also be adjusted by moving the shell connectors **(X)**. After altering the circumference, it may be necessary to adjust the tensioning strap system.

Note: The plastic can be cut with standard leather shears. Make sure that you smooth off the cut edge.

^{*} Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).

¹ A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of supports that are relevant to you.

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