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en Instructions for Use

Product description and performance characteristics
Suprasorb® F Protect Film Wound Dressing is a cover film that provides wound protection by virtue of its material composition. The thin adhesive-coated polyurethane film provides both mechanical protection from friction and protection from water and bacteria. At the same time, the film allows any water vapour that forms under the dressing to escape.
Due to its adhesive strength, Suprasorb® F Protect Film Wound Dressing is suitable for the fixation of primary dressings (e.g. Suprasorb® X, Suprasorb® X+PHMB, Wound Dressings).

Product composition
Polyethylene, polyurethane, polyacrylate, paper coated with polydimethylsiloxane, polyester coated with polyacrylate

Intended purpose
Suprasorb® F Protect Film Wound Dressing is intended for covering and protecting superficial wounds, provided that they are lightly exuding. Suprasorb® F Protect Film Wound Dressing can also be used for fixation in wound care.
Suprasorb® F Protect Film Wound Dressing is also suitable for supporting prophylaxis of skin areas at risk of pressure ulcers.

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 superficial²

¹ Light secretion of fluid

² Injury to the skin without involvement of deeper tissue structures (e.g. abrasion)

Contraindications

- Known allergy and / or hypersensitivity to any of the product components.

Side effects
In individual cases skin irritations and/or allergies may occur.

Warnings and Precautions

- Do not use if package is damaged or accidentally before applying the dressing. Suprasorb® F Protect Film Wound Dressing adheres best to dry skin.
- The product is used to cover and protect superficial wounds. If any signs of infection (extreme redness, marked local increased warmth, odour or pain) occur at the wound site or you are unsure about the application of the dressing, contact your doctor or pharmacist.
- The product is used to cover and protect superficial wounds. If any signs of infection (extreme redness, marked local increased warmth, odour or pain) occur at the wound site or you are unsure about the application of the dressing, contact your doctor or pharmacist.
- Not suitable for the use in negative pressure wound therapy.

Application instructions
Suprasorb® F Protect Film Wound Dressing, sterile is available in different sizes. Ensure that you select the size suitable for its application.

The adhesive film wound dressing should generally be applied approx. 2 cm over the wound edge/the primary dressing being fixed.

1. Preparation
Cleanse the wound/application site and pat the edges dry before applying the dressing. Suprasorb® F Protect Film Wound Dressing adheres best to dry skin.

2. Application

1. Prior to use, visually inspect the sterile barrier system for breaches to the packaging integrity. It should be opened before use, and the dressing should be applied and remove the product (see Fig. 1).
2. Turn the Suprasorb® F Protect Film Wound Dressing so that the printed side faces you. Hold the grip tabs between thumbs and index fingers and pull completely apart (Fig. 2).
3. Turn the product 180° so that the adhesive surface faces the patient/wound (Fig. 3).
4. Place the product centrally over the wound (the adhesive surface should be applied to the wound site, especially at the edges) (Fig. 4).
5. Then hold the grip tabs between thumbs and index fingers and pull them off completely together with the support film (Fig. 5).

Removal
To remove the product with as little pain as possible, the bond between the adhesive and the skin can be released by overstretching the plastic film. When doing so, stretch the film wound dressing as much as possible parallel to the skin (Fig. 6).

3. Changing the dressing
The interval at which Suprasorb® F Protect Film Wound Dressing must be changed are determined by the treating healthcare professionals. These depend on the patient's individual skin condition and on the underlying medical findings. Dressings that are soiled or no longer adhering properly should be replaced.

4. Disposal
In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 18 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance - AVV). Recyclable packaging should be recycled according to the relevant national recycling systems. Please dispose according to local regulations.

General instructions
In case of serious problems (serious incidents), contact the manufacturer and the responsible health authorities.

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