Suprasorb® P Finger/Toe

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en Instructions for Use

Product Description

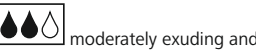
Suprasorb® P Finger/Toe PU Foam dressing, non-adhesive is a sterile and single use wound dressing. The dressing has a triple-layered construction that combines an absorbent polyurethane foam between a perforated non-adherent wound contact layer and an outer waterproof film layer, which has high permeability to allow effective vapor transfer.

Mode of Action


Suprasorb® P Finger/Toe PU Foam dressing, non-adhesive can effectively protect the extremities, absorb wound exudates and maintain a moist wound environment.

Intended purpose

Suprasorb® P Finger/Toe PU Foam dressing, non-adhesive may be used to treat chronic and acute wounds, provided they are

moderately exuding and

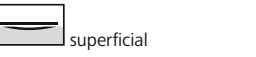superficial

Indications

- Pressure ulcers
- Diabetic foot ulcers
- Surgical wounds
- Donor sites
- Abrasions
- partial superficial burn

The anatomical shape makes it easy to treat hard-to-reach areas such as fingers or toes.

Contraindications

Do not use the product on patients with known sensitivity to any of the product components.

Application instructions

1. Preparation

Clean the wound carefully in accordance with current clinical standards. The skin surrounding the wound should be clean and dry.

Before opening the sterile barrier visually inspect the packaging for obvious damages.

2. Applying the wound dressing

- a) The dressing should overlap the wound edges by approximately 2cm. Cut to size if necessary.
- b) Open the cavity of the dressing completely, gently put Finger or Toe in from the opening side and adjust the dressing to make sure the wound is wrapped well.
- c) Secure Suprasorb® P Finger/Toe PU Foam dressing, non-adhesive with a suitable secondary dressing, such as adhesive tape.

3. Dressing change

Remove the secondary dressing in accordance with current clinical standards. Lift the Suprasorb® P Finger/Toe PU Foam dressing, non-adhesive away from the wound. Suprasorb® P Finger/Toe PU Foam dressing, non-adhesive can stay in place up to 7 days or until the exudate is visible and approaches 2cm from the edge of the foam pad, whichever is sooner. During the early stage of wound management, this product should be inspected frequently.

Precautions

- If you are unsure of the severity of the wound or condition, consult a healthcare professional prior to the first use of this product.
- In case of signs of infection (e.g. excessive pain or swelling), consult a healthcare professional for adequate infection treatment.
- If reddening or sensitization occur, discontinue use.
- Do not use this product with oxidizing agents such as hydrogen peroxide or hypochlorite solutions.
- This product is a single use product. If used on more than one patient, cross-contamination or infection may occur. Opening the dressing pack compromises the sterile barrier therefore any unused dressing should not be retained for use at a later date.
- Single product cannot be stayed on wound for more than 7 days.

Note

Any serious incident that occurs in relation to the use of Suprasorb® P Finger/Toe PU Foam dressing, non-adhesive, should be reported to the authorized representative in the European Community.

Storage and disposal

1. Please store the product in a clean and dry place in specified temperature range and away from sunlight.
2. Disposal should be handled according to local environmental procedures.

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